
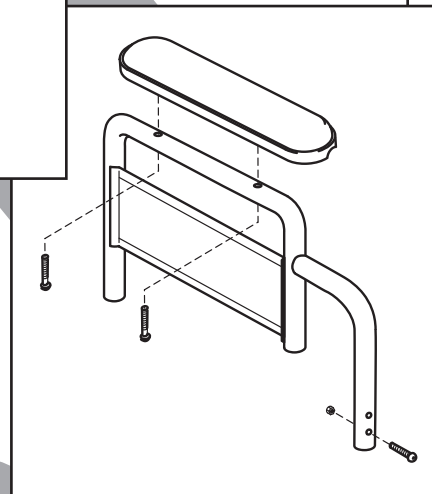
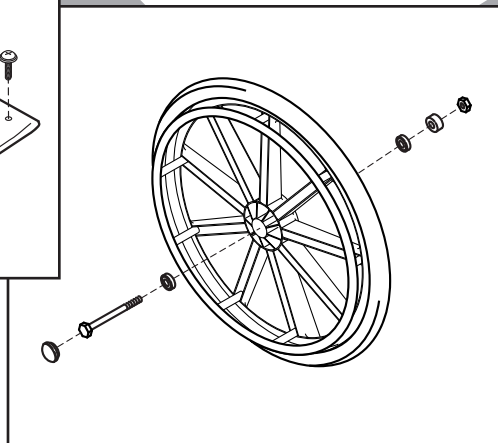


Traveler*[®]** ***HTC **PARTS CATALOG**

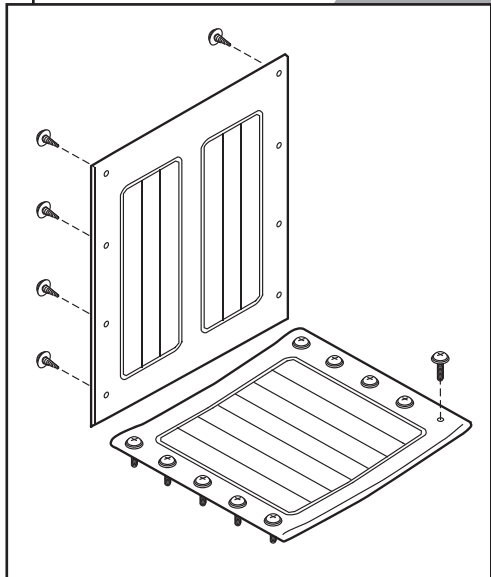

Everest & Jennings[®]

CONTENTS

SIDEFAME AND CROSSBRACE3

ARM, DETACHABLE4

UPHOLSTERY4

REAR WHEEL5

CASTER.....5

FRONT RIGGING, DETACHABLE6

ACCESSORIES7

TO ORDER

Tel: 770-368-4700 • Fax: 770-368-2386


Returns: Parts may not be returned without prior authorization from GF Health Products, Inc. Custom parts cannot be returned for credit. Please refer to GF Health Products, Inc. Terms and Conditions for further information.

GF Health Products, Inc. is not responsible for typographical errors. All illustrations, specifications, packaging and warranties contained in this catalog are based on the latest product information available at the time of printing. The most current product information, including the most current revision of this catalog, can be found online at www.grahamfield.com.

Everest & Jennings, E&J, Traveler, and Graham-Field are registered trademarks of GF Health Products, Inc.

© 2012, GF Health Products, Inc., All Rights Reserved

SIDEFRAME AND CROSSBRACE

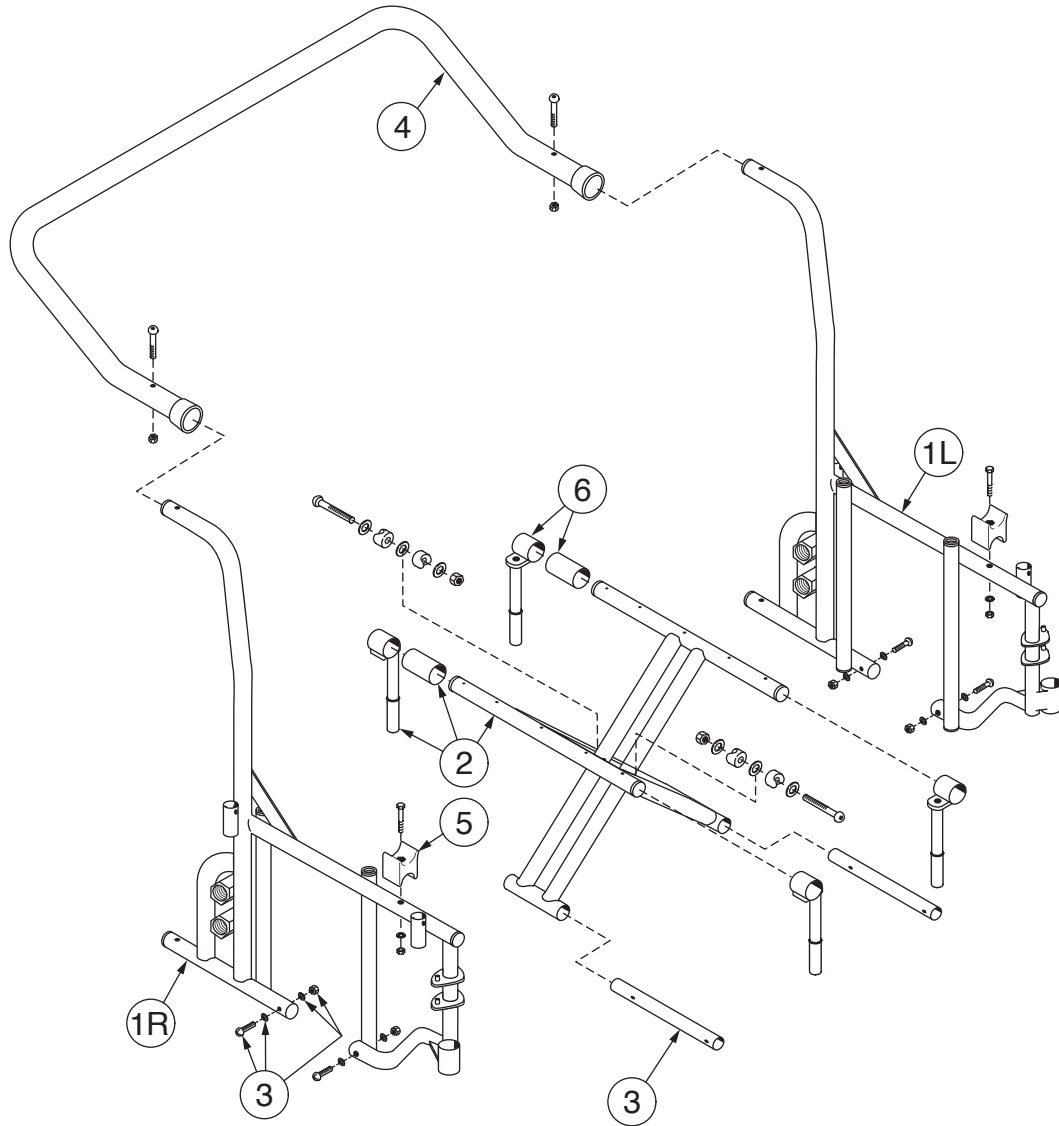

1 Sideframe, all widths

Right 90763601
Left 90763602

2 Crossbrace, pair

Includes slides, center bolt and nut

20" W 907634C1
22" W 907634C2
24" W 907634C3

3 Bottom inner rail kit

Includes bottom inner rail (pair),
screw and nut that hold bottom
inner rail (4 each), and sleeve (pair) ... 90763503

4 Push handle

Includes hardware

20" W 907636B1
22" W 907636B2
24" W 907636B3


5 Seat rail cradle, screw, washer and nut (pair)

90763403

6 Seat slide and seat rail sleeve (2 each)


90763404

ARM, DETACHABLE

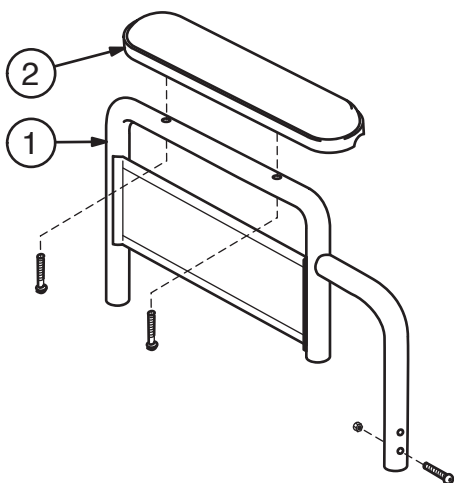

- 1 Arm, detachable desk length
fixed-height, pair
Includes right and left arm with
side panel and mounting hardware
(does not include arm pad) 90763620
- 2 Arm pad, desk length, pair
Includes two black arm pads
and mounting screws 90763022

UPHOLSTERY

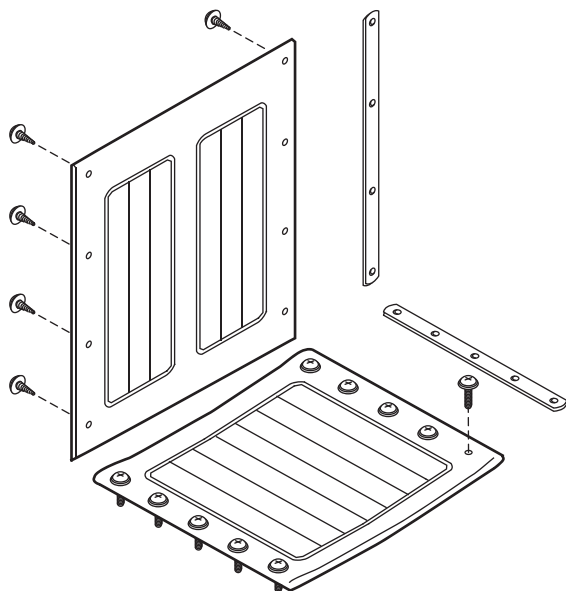


Back upholstery, black leatherette,
with built-in chart holder
Includes mounting strips and screws

20" W.....	907636A1
22" W.....	907636A2
24" W.....	907637A3

Seat upholstery, black leatherette
Includes mounting strips and screws


20" W x 18" D.....	907634B1
22" W x 18" D.....	907634B2
24" W x 18" D.....	907634B3

REAR WHEEL

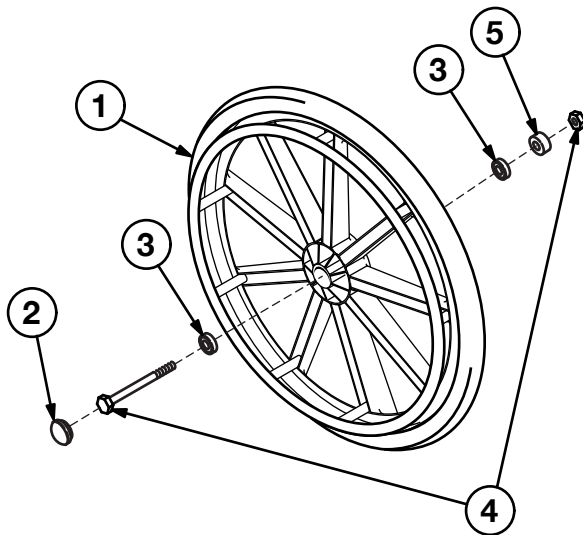


- 1 Rear wheel with handrim, 24" x 1",
polyurethane tire
Includes hub cap and bearing;
does not include axle or nut..... 90763450
- 2 Hub cap (10 each)..... 90763451
- 3 Wheel bearing (10 each) 90763452
- 4 Rear axle and nut (10 each)..... 90763453
- 5 Spacer (10 each)..... 90763454

CASTER

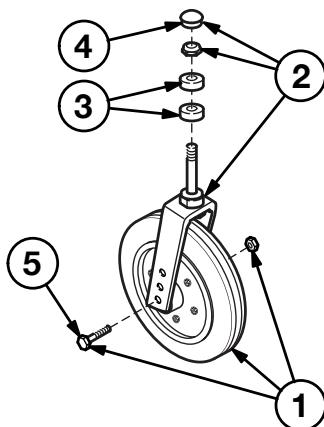

- Complete caster (fork and wheel),
8" x 2" solid urethane tire 90763440
- 1 Caster wheel, axle and nut 90763441
- 2 Caster fork, stem nut and dust cap 90763042
- 3 Caster stem bearing (10 each) 90763043
- 4 Caster dust cap (10 each)..... 90763044
- 5 Caster axle and nut (10 each)..... 90763045


FRONT RIGGING, DETACHABLE

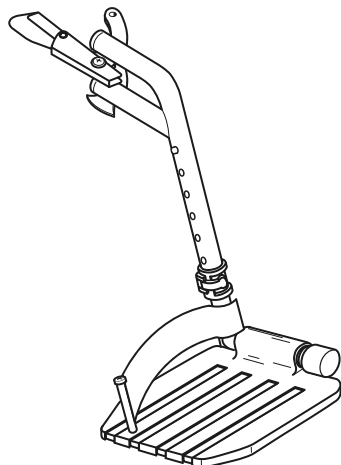

Optional footrest, pair, all widths
Includes right and left hanger with release
and aluminum footplate with extension
and heel loop 90763430


Aluminum footplate (pair)
(Fits all front rigging) 90763433

Anti-Theft nut (set of 4)
(Fits all front rigging) 90763630

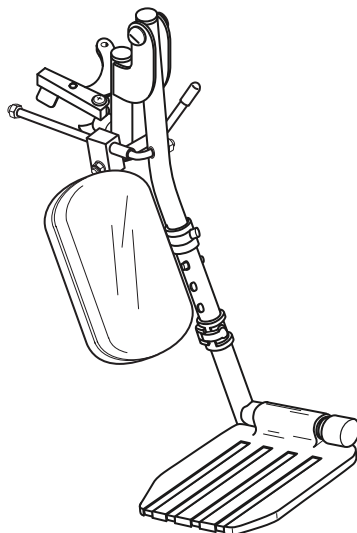

Low-pivot elevating legrest, pair, all widths
Includes right and left hanger with release,
aluminum footplate with extension, and
legrest panel (does not include
heel loop) 90763431

Elevating legrest panel kit
(Fits all elevating legrests)
Includes 2 black padded panels
and hardware 90763032

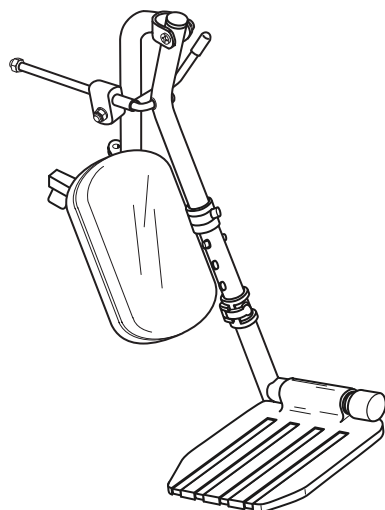

High-pivot elevating legrest, pair, all widths
Includes right and left hanger with release,
aluminum footplate with extension, and
legrest panel (does not include
heel loop) 90763435

Traveler HTC accessories (not shown)

1	Heel loop, pair.....	90763064
2	Attendant wheel lock, pair (Includes hardware)	90763640
3	Rubber tip, wheel lock (10 each).....	90763063
4	Positioning belt (metal buckle)	90133125
5	O2-IV holder	B10320F
6	O2 cylinder holder.....	B10325
7	Anti-Tippers, pair (Includes hardware)	90763650
8	Anti-Theft pole (includes hardware)...	90763660

Everest & Jennings®

**GF Health Products, Inc.
2935 Northeast Parkway
Atlanta, Georgia 30360 USA**

tel: 770-368-4700

fax: 770-368-2386



GRAHAM-FIELD

www.grahamfield.com

**Traveler HTC Parts Catalog
GF1200119RevB14**