

# GF8905-1A: DELUXE TILT OVERBED TABLE ASSEMBLY & OPERATION INSTRUCTIONS

# PLEASE SAVE THESE INSTRUCTIONS FOR FUTURE USE


Info: The most current version of these instructions can be found online at <u>www.grahamfield.com</u>.

Tools needed: Phillips screwdriver, 9/16 hex wrench, 5/16 hex wrench

- ⚠ WARNING: Important! Read and understand these instructions before assembling or using the GF8905-1A Overbed Table. If you do not understand any part of these instructions, contact a healthcare professional for direction in the use of this product. If the Overbed Table is not properly assembled and adjusted, personal injury and/or damage to the Overbed Table could result.
- ⚠ WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and/or damage to Overbed Table.

### UNPACKING

- 1. Check for obvious damage to the carton or its contents. If damage is evident, notify the carrier and your Graham-Field authorized distributor.
- 2. Remove all loose packing from the box. Carefully remove all the components from the carton. Components: Table top, mast, 6 machine screws, base with 4 casters, 2 hex head bolts, 2 flat washers, and 2 metal spacers.

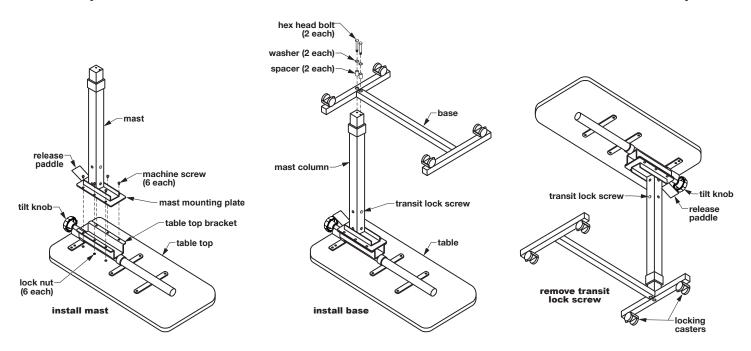

# **ASSEMBLY (SEE PICTURES ABOVE)**

- $\triangle$  WARNING: Do not remove the transit lock screw from the mast before the table is completely assembled; the table top could rise suddenly and cause serious personal injury.
- $ilde{m{m{m{m{m{m{M}}}}}}$  WARNING: Do not disassemble the mast; this could cause serious personal injury.
- 1. Place the table top upside down on work surface.
- 2. Position the mast above the table top, oriented as shown at left above. Place mast mounting plate on table top, matching mast mounting plate holes with table top bracket holes.
- 3. Mount mast to table top: Install machine screws through mast mounting plate holes and table top bracket holes. Install nuts on screws. Tighten.
- 4. Place the base upside down above the mast, oriented as shown at center above.
- 5. Install the two hex head bolts through the washers, spacers, and base, and into the end of the mast column as shown. Tighten the bolts.

GF8905-1A-INS-LAB-RevC14

- 6. Turn the table upright and place it on the floor, as shown at far right on previous page. While holding table top down with one hand, carefully — table could rise suddenly — remove the transit lock screw with the other hand. Discard transit lock screw.
- $ilde{\Lambda}$  WARNING: Ensure that the table is assembled as described, and that all components are securely attached, before use.

## **OPERATION INSTRUCTIONS**

- $\triangle$  WARNING: Do not lean on the suspended end of the table top; this may cause the table to tip.
- ♠ WARNING: The table supports up to 30 lb of dispersed weight. Do not exceed this limit.

To raise the table top: apply light pressure upward on the bottom of the table top and stop when the table top has reached the desired position.

To lower the table top: squeeze the release paddle upward while pushing the table top down; release the paddle when the table top has reached the desired position.

To tilt the table top in either direction: pull out the tilt knob, tilt the table top, and release the knob when the table top is in the desired position.

To lock and unlock the casters: the two mast-end casters have step-on locks. Step on locks to lock casters; pull locks up to unlock.

# **MAINTENANCE / CLEANING**

Check the fit and tightness frequently of all nuts and bolts to ensure that the table is securely assembled.

# ▲ NOTICE: Do not use cleansers that may damage the table's finish.

To clean, gently wipe the table with a soft, clean, damp cloth. To wash more thoroughly, clean with mild soap and water. Dry table with a clean, soft cloth.

# LIMITED WARRANTY

#### **SCOPE OF WARRANTY**

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, this product is warranted for three (3) years, and the normal wear components such as tips, wheels/casters, and handgrips are warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

### **OBTAINING WARRANTY SERVICE**

A GF Customer Service Representative must authorize warranty service. Please contact the GF Customer Service department by calling 678-291-3207, sending a fax request to 770-368-2386 or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

#### ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED HIS WARRANTI I SUF 3 OULT WARRANT I AND 3 IN LIEU OF ALL OTHER WARRANTIES, EAR RESSORT IN HIS LIEU WARRANTIES OF ANY RIND ROLL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS SHOWN TO THE CONFORM TO THE MODEL OR SAMPLE WAS SHOWN TO THE CONFORM TO THE MODEL OR SAMPLE WAS SHOWN TO THE CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS, GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

#### **NOTES:**

- Additional terms and conditions may apply.
- Freight claims must be notated on the Bill of Lading and must be made with immediacy. The ICC regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim
- 3) Claims for any short shipment must be made within thirty (30) days of the invoice date.



GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at www.grahamfield.com Graham-Field and Lumex are registered trademarks of GF Health Products, Inc.

© 2006 GF Health Products, Inc. • 2935 Northeast Parkway, Atlanta, GA 30360 • tel 770-368-4700 • fax 770-368-2386 • Made in China

GF8905-1A-INS-LAB-RevC14