



549 SERIES SUPPORTO TRUSS



INSTRUCTIONS FOR USE

CAUTION: This product contains natural rubber latex which may cause allergic reactions.

Read these instructions before using the Pipeer Supporto Truss. Save these instructions for future reference.

The most current version of these instructions can be found online at www.grahamfield.com.

Safety

- ⚠ WARNING: DO NOT use this product without proper instruction from a healthcare professional.**
- ⚠ WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**

Intended use

The intended use of the Supporto Truss is the treatment of reducible inguinal hernia and athletic support. Another undergarment is not needed while wearing Supporto Truss.

Application

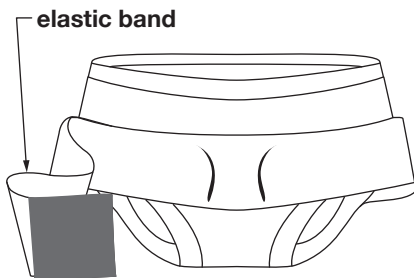


fig. 1

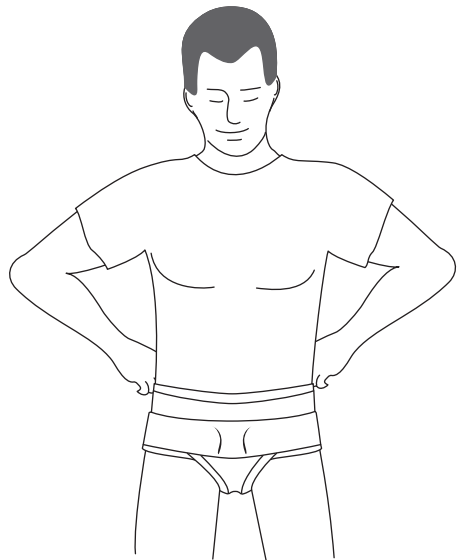


fig. 2

1. Pull apart the elastic band. Leave unattached.
2. If rupture is protruding, lie down. Reduce rupture size to normal by using the palm of your hand to gently push rupture upward and in. If necessary, cough to locate exact position of rupture opening.

3. Step into the Supporto Truss as you would any briefs or undershorts. Pull up into proper fitting position. There are two pads to provide balanced support for reducible inguinal hernia. Center pads on exact rupture opening.
4. Stand up straight. Take one end of the elastic band in each hand and pull out and around your back to attach the hook and loop closure firmly. Adjust until the fit is comfortably snug. **ENSURE THE PAD IS PLACED AT THE POINT OF RUPTURE.**

Removal

To remove Supporto Truss, unfasten the elastic band hook and loop closure and step out of Supporto Truss.

Cleaning and Maintenance

Machine wash Supporto Truss in warm water, gentle cycle, use mild soap. Tumble dry low, slow, short cycle.

▲ **NOTICE: DO NOT use detergent or bleach to clean Supporto Truss. DO NOT iron Supporto Truss.**

Thirty-Day Limited Warranty

GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the Pipeer 549 Series Supporto Truss is warranted for thirty days. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component, in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling 678-291-3207, sending a fax request to 770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.



**Manufactured by:
GF Health Products, Inc.
Atlanta, Georgia 30360**

Proudly made in the U.S.A.

**GF Health Products, Inc.
2935 Northeast Parkway
Atlanta, GA 30360
telephone: 770-368-4700
fax: 770-368-2386**

www.grahamfield.com

© 2012 GF Health Products, Inc.

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com. **Graham-Field** and **Pipeer** are registered trademarks of GF Health Products, Inc.