
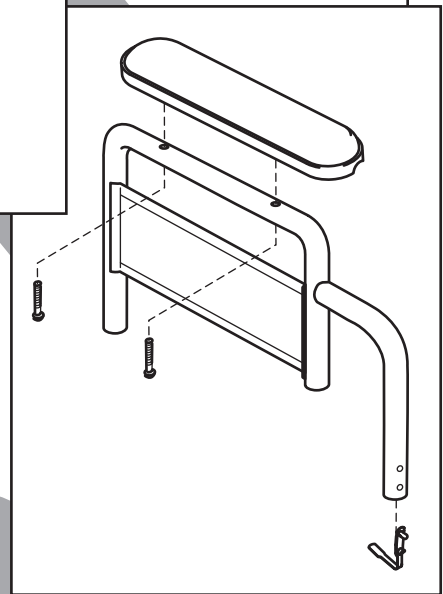
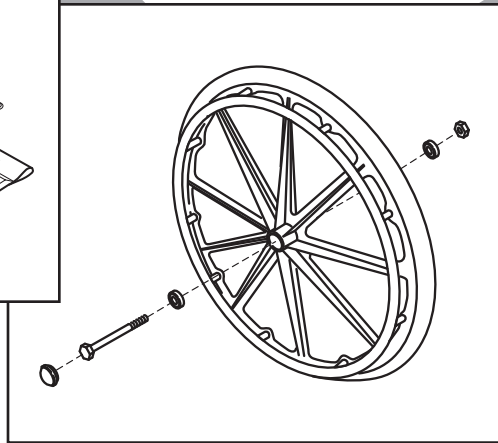


Advantage[®]

PARTS CATALOG

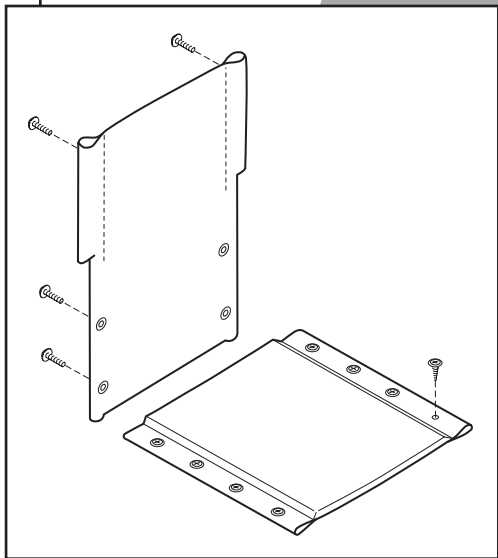

Everest & Jennings[®]

CONTENTS

SIDEFRAME (FIXED ARM), CROSSBRACE, ARM PAD KIT3

SIDEFRAME (DETACHABLE ARM), CROSSBRACE.....4

ARM, DETACHABLE5

CASTER.....5

REAR WHEEL5

FRONT RIGGING, DETACHABLE6

UPHOLSTERY7


ACCESSORIES7

TO ORDER

Tel: 770-368-4700 • Fax: 770-368-2386

Returns: Parts may not be returned without prior authorization from GF Health Products, Inc. Custom parts cannot be returned for credit. Please refer to GF Health Products, Inc. [Terms and Conditions](#) for further information.

SIDEFRAME (FIXED ARM), CROSSBRACE, ARM PAD KIT

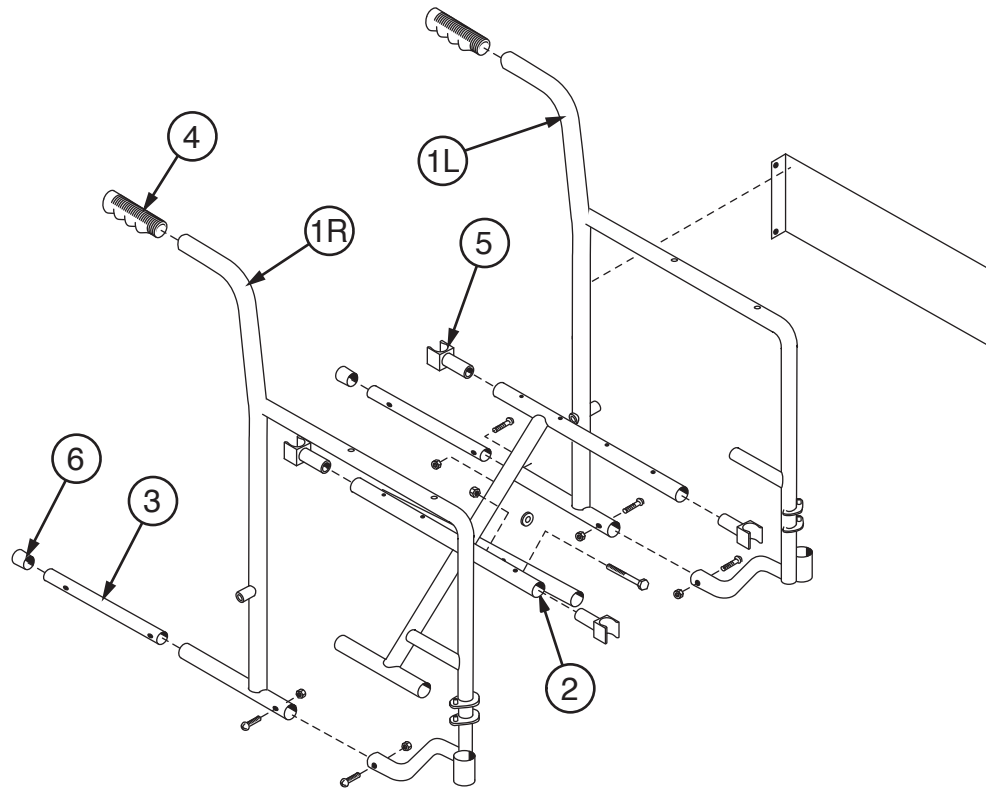

1 Sideframe, fixed arm:
Includes side panel

Right 90763501
Left 90763502

2 Crossbrace, pair:
Includes center bolt and nut


16" W 907635H1
18" W 907635H2

3 Bottom inner rail kit: Includes
bottom inner rail (2 each) and
screw and nut that hold bottom
inner rail (4 each)..... 90763503

4 Grip (10 each) 90763005


5 Seat rail guide (4 each) 90763004

6 Tipping sleeve (10 each)..... 90763006

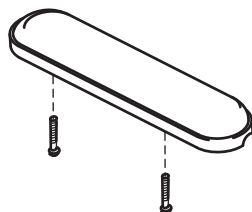

Arm pad kit: Includes 2 black
full-length arm pads and
mounting screw 90763524

SIDEFAME (DETACHABLE ARM), CROSSBRACE

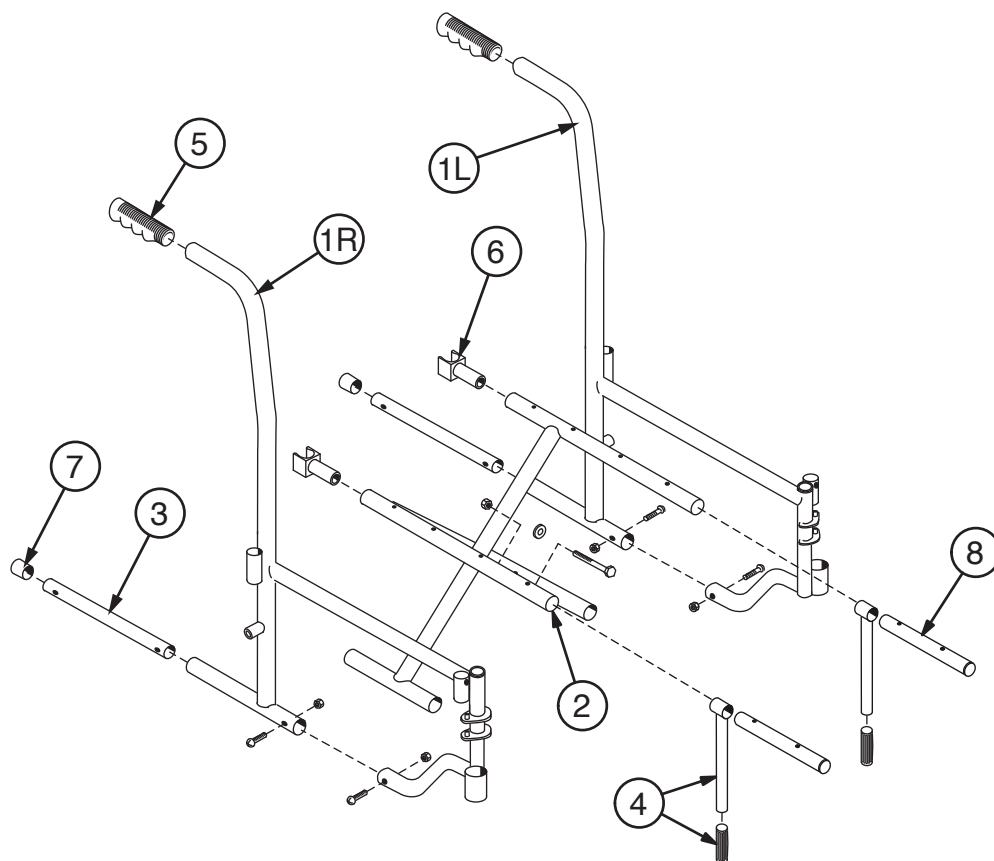

1 Sideframe, detachable arm

Right 90763511
Left 90763512

2 Crossbrace, pair: Includes center bolt and nut

16" W 907635C1
18" W 907635C2
20" W 907635C3

3 Bottom inner rail kit: Includes bottom inner rail (2 each); screw and nut that hold bottom inner rail (4 each); and sleeve (2 each)

..... 90763503

4 Seat slide kit:

Single telescopic: Includes seat slide (2 each) and bushing (2 each)

16" and 18" W 90763513

Double telescopic: Includes seat slide (2 each)

20" W 90763517


5 Grip (10 each) 90763005

6 Seat rail guide, rear (2 each) 90763015

7 Tipping sleeve (10 each) 90763006

8 Inner seat rail (2 each) 90763514

ARM, DETACHABLE

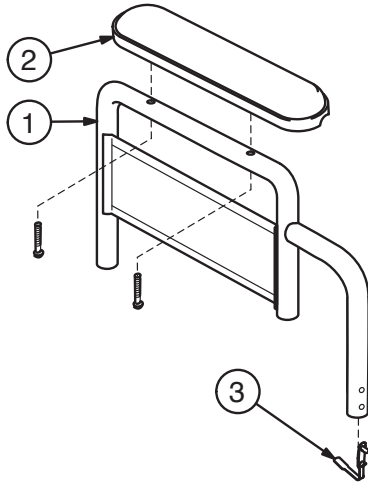

- 1 Arm, detachable fixed-height, pair:**
Includes right and left arm with side panel and spring lock (does not include arm pad)


Desk length..... 90763020
Full length..... 90763021

- 2 Arm pad, pair:** Includes two black arm pads and mounting screws

Desk length..... 90763022
Full length..... 90763023

- 3 Spring lock (10 each) 90763024**


CASTER

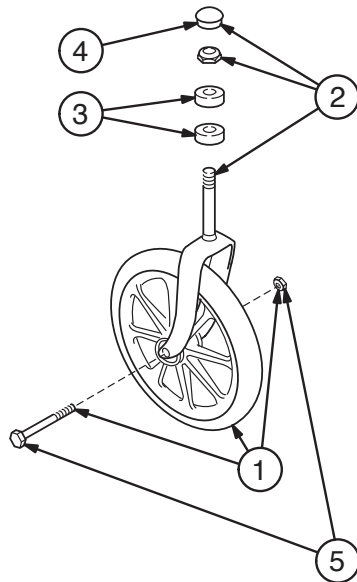

Complete caster (fork and wheel),
8" x 1" solid rubber tire..... 90763540

- 1 Caster wheel, axle and nut 90763541**
2 Caster fork, stem nut and dust cap 90763542
3 Caster stem bearing (10 each) 90763543
4 Caster dust cap (10 each)..... 90763544
5 Caster axle and nut (10 each)..... 90763545

REAR WHEEL

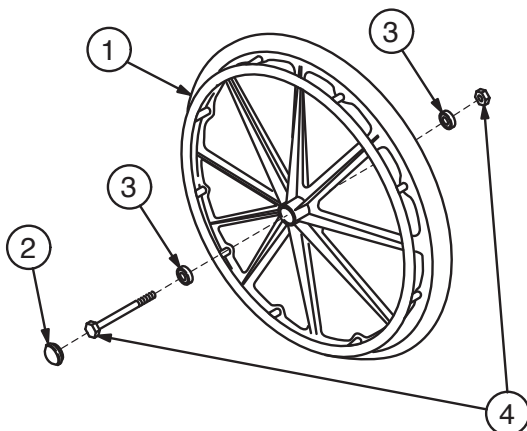

- 1 Rear wheel with handrim, 24" x 1", solid rubber tire: Includes hub cap and bearing; does not include axle or nut 90763550**


- 2 Hub cap (10 each)..... 90763051**

- 3 Wheel bearing (10 each) 90763552**

- 4 Rear axle and nut (10 each)**


Fixed-arm sideframe 90763553
Detachable-arm sideframe 90763554

FRONT RIGGING, DETACHABLE

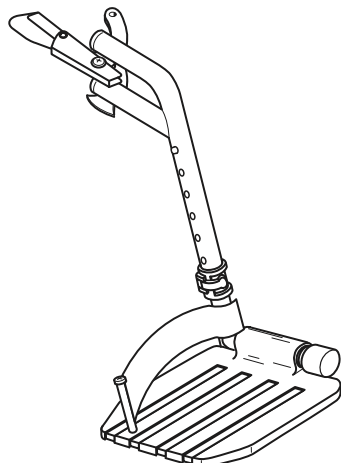


Footrest, pair, all widths
Includes right and left hanger with release
and composite footplate with extension
and heel loop 90763030

Composite footplate, pair
(Fits all front rigging) 90763033

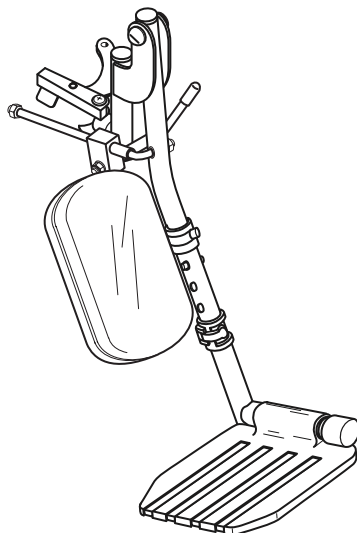


Low-pivot elevating legrest, pair, all widths
Includes right and left hanger with release,
composite footplate with extension, and
legrest panel (does not include
heel loop) 90763031

Elevating legrest panel kit
(Fits all elevating legrests)
Includes 2 black padded panels
and hardware 90763032

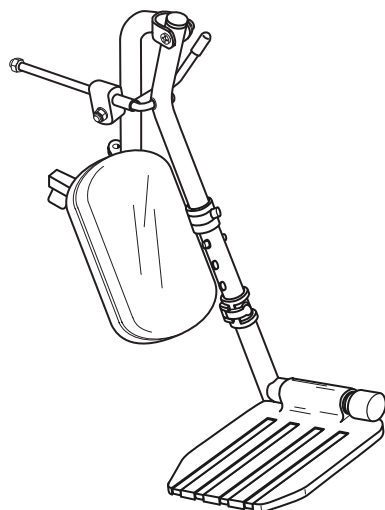

High-pivot elevating legrest, pair, all widths
Includes right and left hanger with release,
aluminum footplate with extension, and
legrest panel (does not include
heel loop) 90763435

UPHOLSTERY

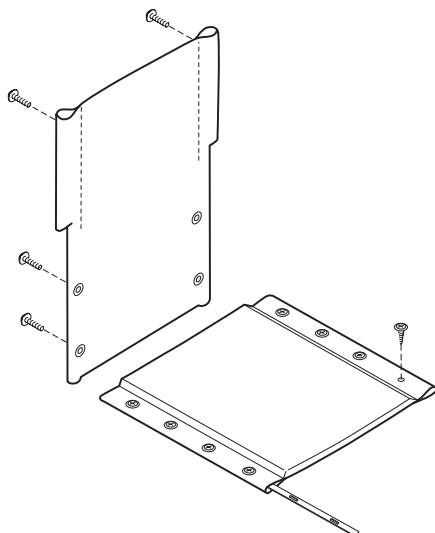

Back upholstery, black, with built-in chart holder: Includes hardware

Vinyl, fixed arm, 16" wide	907635F1
Vinyl, fixed arm, 18" wide	907635F2
Vinyl, detachable arm, 16" wide.....	907635E1
Vinyl, detachable arm, 18" wide.....	907635E2
Vinyl, detachable arm, 20" wide.....	907635E3
Nylon, fixed arm, 16" wide	907635D1
Nylon, fixed arm, 18" wide	907635D2
Nylon, detachable arm, 16" wide ...	907635A1
Nylon, detachable arm, 18" wide ...	907635A2
Nylon, detachable arm, 20" wide ...	907635A3

Seat upholstery, black: Includes mounting strips and hardware

Vinyl, 16" wide	907635G1
Vinyl, 18" wide	907635G2
Vinyl, 20" wide	907635G3
Nylon, 16" wide	907635B1
Nylon, 18" wide	907635B2
Nylon, 20" wide	907635B3

ACCESSORIES

Advantage accessories (not shown)

1 Heel loop, pair.....	90763064
2 Wheel lock, pair: Includes rubber tips	
Fixed-arm sideframe	
Push to lock	90763070
Pull to lock	90763071
Detachable-arm sideframe	
Push to lock	90763060
Pull to lock	90763061
3 Rubber tip, wheel lock (10 each).....	90763063
4 Wheel lock extension, pair	90763062
5 Positioning belt (metal buckle)	92133040
6 Anti-Fold bar	92523850
7 O2-IV holder	B10320F
8 O2 cylinder holder.....	B10325
9 Anti-Tippers, pair:	
Fixed-arm sideframe	90763566
Detachable-arm sideframe	90763565



1.770.368.4700

Information contained herein is subject to change.

The most current and complete product information can be found on our website.

www.grahamfield.com



© 2009, GF Health Products, Inc. All Rights Reserved.

Advantage, E&J, Everest & Jennings, and Graham-Field are trademarks of GF Health Products, Inc.
GF Health Products, Inc. is an ISO 13485 Certified Company.

Advantage Parts Catalog
GF0900221RevG16