

Info: The most current version of these instructions can be found online at www.grahamfield.com.

Tools required: Phillips or blade screwdriver

- ⚠ WARNING:** Important! Read and understand these instructions before installing or using the Toilet Seat Riser. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the toilet seat riser is not properly assembled and installed, personal injury and damage to the toilet seat riser could result.
- ⚠ WARNING:** If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the toilet seat riser.
- ⚠ WARNING:** This product has a weight limit of 300 lb, EVENLY DISTRIBUTED.
- ⚠ WARNING:** Notice for California Customers - California Proposition 65 **WARNING:** This product can expose you to chemicals including DEHP which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to www.P65Warnings.ca.gov/product.
- ⚠ WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.

INTENDED USE



6482R-2 assembled





6482ER-2 assembled

The toilet seat riser is intended for use by those who have difficulty sitting down on or standing up from the toilet.

INSTALLATION

1. Remove toilet seat riser, hardware, and armrests from carton and protective packaging.
2. Install the armrests using the following steps:
 - a. Identify the right and left armrests by the **R** (Right) and **L** (Left) stamped into armrest ends.
 - b. Note that each toilet seat riser armrest socket is also stamped with an **R** or **L**, as shown at right. Insert each armrest marked with **R** or **L** into the socket with the corresponding letter.
 - c. Push each armrest into its socket until it clicks into place.
 - d. Ensure locking buttons protrude fully from holes. Pull on armrests slightly to test proper installation. If loose, repeat step c.
3. At the toilet, lift the cover plates at top rear toilet bowl rim, exposing the bolts holding the seat and lid to the toilet bowl.
4. Use a Phillips or blade screwdriver, whichever is appropriate, to unscrew toilet seat bolts by turning counterclockwise.
5. Remove the toilet seat and lid from the toilet bowl and set them aside.
6. Place the toilet seat riser over the toilet bowl rim so that the toilet seat riser mounting holes align with the toilet rim holes.
7. Place the toilet seat and lid on top of the toilet seat riser so that the seat/lid holes align with the riser and rim holes.
8. Install a washer on one of the long bolts included in the box and insert it through the top of the toilet seat lid, toilet seat riser, and toilet bowl rim.
9. Place a washer and wing nut on the end of the long bolt that protrudes through the bottom of the toilet bowl.
10. Use a Phillips or blade screwdriver, whichever is appropriate, to turn the bolt clockwise until toilet seat riser and toilet seat lid are snug. Do not overtighten.
11. The toilet seat riser should now be securely installed on the toilet bowl. To ensure that you have a proper fit, intentionally shake the toilet seat riser to ensure proper placement and fit. The toilet seat riser should not move; if it does, re-install.

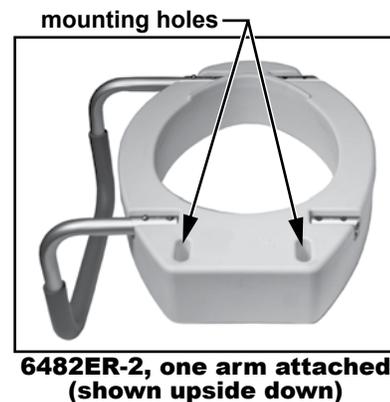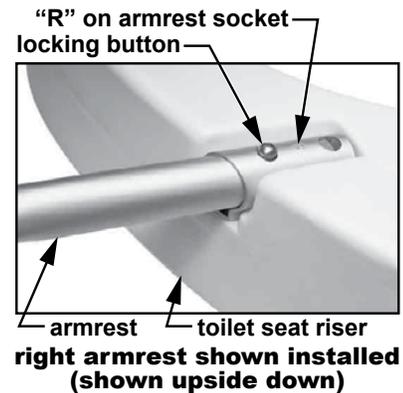

OPERATION



6482R-2 assembled



6482ER-2 assembled

- ⚠ **WARNING:** Always ensure the toilet seat riser is securely installed before use.
- ⚠ **WARNING:** Users with limited physical capabilities should be supervised or assisted when using toilet seat riser.

MAINTENANCE / CLEANING

- ⚠ **WARNING:** Check the toilet seat riser weekly for signs of damage or cracking. If damage is found, **DO NOT** use the toilet seat riser. Contact your GF authorized distributor immediately.

Loosen, retighten, and check for secure fit of toilet seat riser weekly. To clean toilet seat riser, wash with soap and water or a mild household detergent and rinse with clear water. Ensure the toilet seat riser is rinsed well before use.

If moisture accumulates inside toilet seat riser, do the following:

1. Remove toilet seat riser from toilet bowl.
2. Turn toilet seat riser upside down and tilt toward drain opening.
3. Rotate until drain opening is facing downward and moisture drains from toilet seat riser.
4. Allow toilet seat riser to air dry for several hours.

- ▲ **NOTICE:** Never use abrasive cleaners or scouring pads on any part of the toilet seat riser.

LIMITED WARRANTY

SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, this product is warranted for six (6) months. The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1-770-368-4700, sending a fax request to 1-770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.



1.770.368.4700

Information contained herein is subject to change.

The most current and complete product information can be found on our website.

www.grahamfield.com



© 2011, GF Health Products, Inc. All Rights Reserved.
Lumex and Graham-Field are trademarks of GF Health Products, Inc.
GF Health Products, Inc. is an ISO 13485 Certified Company.

Manufactured for GF Health Products, Inc.
Made in China