

*Info: The most current version of these instructions can be found online at [www.grahamfield.com](http://www.grahamfield.com).*

#### SAFETY GUIDELINES — READ BEFORE USE

- ⚠ **WARNING: Important! Read and understand these instructions before installing or using the raised toilet seat. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the raised toilet seat is not properly installed, personal injury and damage to the raised toilet seat could result.**
- ⚠ **WARNING: Users with limited physical capabilities should be supervised or assisted when using raised toilet seat.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the raised toilet seat.**
- ⚠ **WARNING: This product has a maximum weight capacity of 250 lb, EVENLY DISTRIBUTED.**
- ⚠ **WARNING: Notice for California Customers - California Proposition 65 WARNING: This product can expose you to chemicals including DEHP which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to [www.P65Warnings.ca.gov/product](http://www.P65Warnings.ca.gov/product).**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

#### INTENDED USE

The raised toilet seat is intended for use by those who have difficulty sitting down on or standing up from the toilet.

#### INSTALLATION INSTRUCTIONS

- ▲ **NOTICE: This raised toilet seat is intended to fit most standard toilets. However, due to the large variety of toilets on the market, this unit may not fit every model, or could be damaged by certain models of toilet. If your toilet has raised or protruding points or sections on the top or inner ring of the bowl, do not use this raised toilet seat. Certain toilet bowls require the removal of the existing toilet seat and lid for proper fit. When properly installed, the raised toilet seat should sit flat on the top of the bowl lip and against the bowl's inner curved section.**



1. Remove raised toilet seat from carton and protective packaging.
2. Lift lid and seat of toilet.
3. Place raised toilet seat on bowl rim as shown at above right, ensuring that raised toilet seat rests flat on toilet bowl rim. Straighten unit on bowl rim.

#### OPERATION

- ⚠ **WARNING: Always ensure the raised toilet seat is securely installed and fits tightly on toilet bowl before use.**
- ⚠ **WARNING: Users with limited physical capabilities should be supervised or assisted when using raised toilet seat.**

#### MAINTENANCE / CLEANING

- ⚠ **WARNING: Check the raised toilet seat weekly for signs of damage or cracking. If damage is found, DO NOT use the raised toilet seat. Contact your GF authorized distributor immediately.**

Loosen, retighten, and check for secure fit of raised toilet seat weekly. To clean raised toilet seat, wash with soap and water or a mild household detergent and rinse with clear water. Ensure the raised toilet seat is rinsed well before use.

**If moisture accumulates inside raised toilet seat, do the following:**

1. Remove raised toilet seat from toilet bowl.
2. Turn raised toilet seat upside down and tilt toward drain opening.
3. Rotate until drain opening is facing downward and moisture drains from raised toilet seat.
4. Allow raised toilet seat to air dry for several hours.

**▲ NOTICE: Do not use abrasive cleaners or scouring pads on any part of the raised toilet seat.**

**LIMITED WARRANTY**

**SCOPE OF WARRANTY**

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, this product is warranted for six (6) months. The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

**OBTAINING WARRANTY SERVICE**

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1-770-368-4700, sending a fax request to 1-770-368-2386, or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

**EXCLUSIONS**

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

**ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER**

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

**NOTES:**

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.



1.770.368.4700

Information contained herein is subject to change.  
The most current and complete product information can be found on our website.  
[www.grahamfield.com](http://www.grahamfield.com)



© 2005, GF Health Products, Inc. All Rights Reserved.  
Lumex and Graham-Field are trademarks of GF Health Products, Inc.  
GF Health Products, Inc. is an ISO 13485 Certified Company.  
Manufactured for GF Health Products, Inc.  
Made in China