


PLEASE SAVE THESE INSTRUCTIONS FOR FUTURE USE

- ⚠ **WARNING:** Important! Read and understand these instructions before installing or using the O₂ Holder. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the O₂ Holder is not properly assembled and installed, personal injury and damage to the O₂ Holder could result.
- ⚠ **WARNING:** If components are damaged or missing, contact your dealer immediately. **DO NOT** use substitute parts. Use only Everest & Jennings replacement parts. Non-Everest & Jennings replacement parts could cause personal injury and damage to the O₂ Holder.
- ⚠ **WARNING:** Cancer and Reproductive Harm - www.p65warnings.ca.gov.
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.

E&J® TRAVELER®, ADVANTAGE, TRAVELER L3/L4, TRAVELER SE, AND TRAVELER HD: O₂ HOLDER INSTALLATION ON WHEELCHAIR (SEE ILLUSTRATIONS BELOW)

Remove O₂ Holder from carton and protective packaging.

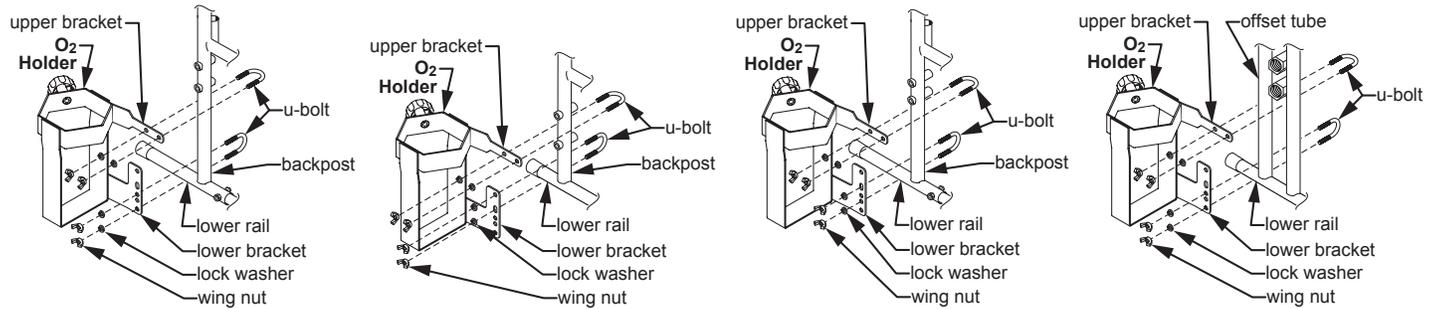





Steps 1-2: Traveler/Advantage

Steps 1-2: Traveler L3/L4

Steps 1-2: Traveler SE

Steps 1-2: Traveler HD

1. Note that left sideframe above is shown from inside. Install O₂ Holder on left-rear sideframe lower rail: Insert U-bolt over rail, then through appropriate lower bracket holes as shown above. Install washers and wing nuts on U-bolt ends, securing O₂ Holder to rail.
2. Install O₂ Holder on sideframe backpost: Insert U-bolt over backpost (except Traveler HD: insert U-bolt over offset tube behind backpost, as shown above), then through upper bracket as shown above. Install washers and wing nuts on U-bolt ends, securing O₂ Holder to backpost/offset tube.
3. Tighten all four wing nuts until O₂ Holder is secure on sideframe as shown at right.

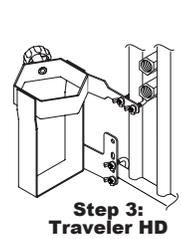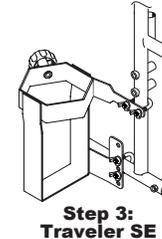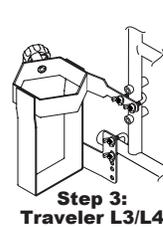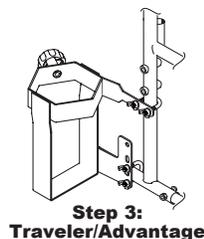


OPERATION

- ⚠ **WARNING:** Ensure the O₂ Holder is installed as described, and all components are securely fastened, before use.

Install O₂ cylinder

Loosen anti-rattle knob as necessary, install cylinder, and tighten knob enough to prevent cylinder from rattling.

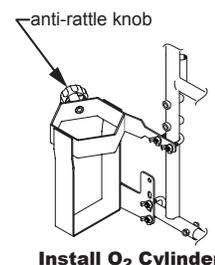

MAINTENANCE

⚠ WARNING: Inspect the O₂ Holder frequently and tighten it as needed to ensure secure installation.

Check O₂ Holder at least once a week to ensure all parts are securely fastened. Wipe O₂ Holder with a soft cloth and dry immediately if exposed to moisture. If necessary, wash with a mild soap and water solution, rinse, and dry with a soft cloth.

▲ NOTICE: Never use abrasive cleaners or scouring pads on any part of the O₂ Holder.

WARRANTY

SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the Original Purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. Original Purchaser is one who purchases this product new and unused from GF or a GF Distributor. This limited warranty shall only apply to defects that are reported within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, this product is warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the Original Purchaser, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1.770.368.4700, sending a fax request to 1.770.368.2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS.

THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document. Some states do not allow the exclusion of certain remedies; in those instances that state's law will control. This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.



1.770.368.4700

Information contained herein is subject to change.

The most current and complete product information can be found on our website.

www.grahamfield.com

Follow us on



© 2011, GF Health Products, Inc. All Rights Reserved.

E&J, Everest & Jennings, Graham-Field, and Traveler are trademarks of GF Health Products, Inc., One Graham-Field Way, Atlanta GA 30340-3140
GF Health Products, Inc. is an ISO 13485 Certified Company. • Manufactured for GF Health Products, Inc. • Made in Taiwan