


- ⚠ **WARNING:** Important: Read and understand these instructions before installing the Adjustable Back. Ensure the Adjustable Back is installed as described in the following paragraphs before use. If the Adjustable Back is not properly assembled, personal injury and/or property damage could result.
- ⚠ **WARNING:** If components are damaged or missing, contact your dealer immediately. **DO NOT** use substitute parts. Use only Everest & Jennings replacement parts. Non-Everest & Jennings replacement parts could cause personal injury and damage to the Adjustable Back.
- ⚠ **WARNING:** Cancer and Reproductive Harm - www.p65warnings.ca.gov.
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.

THE ADJUSTABLE BACK INCLUDES:

Quantity	Description
1	Adjustable Back with back upholstery and 2 backposts
2	1/4" threaded knob
2	1/4" x 1 1/4" hex bolt
2	16" cable tie

ADJUSTABLE BACK INSTALLATION (REFER TO PICTURES BELOW)

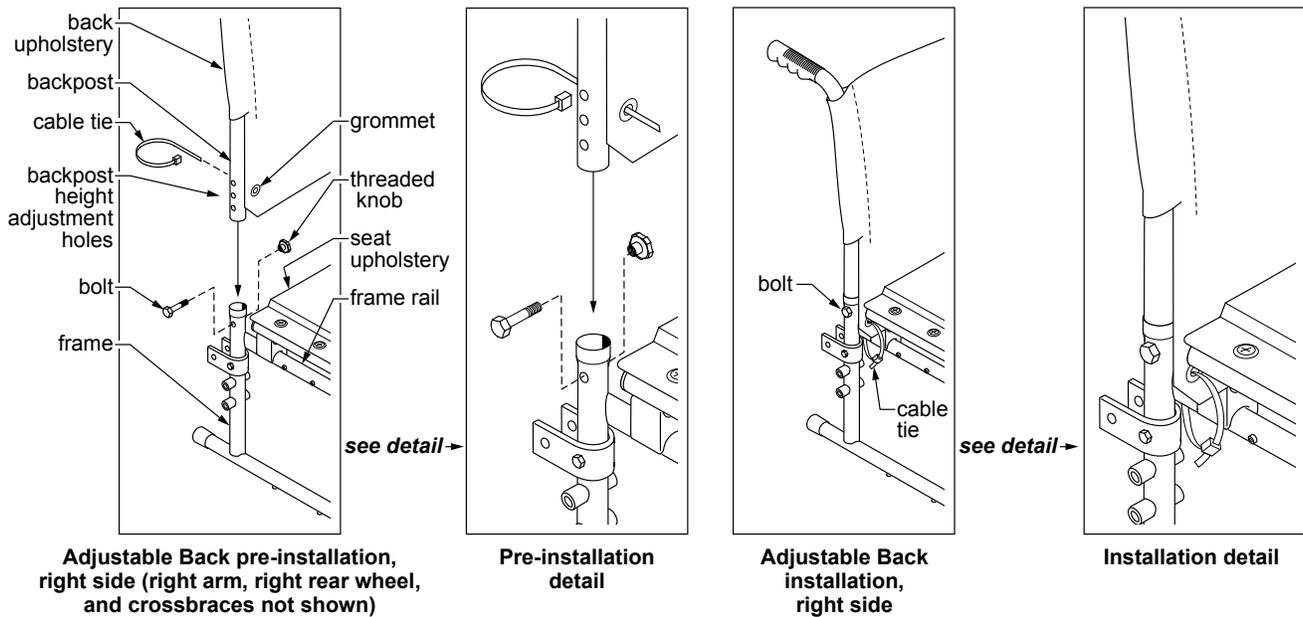

1. Insert backposts into frame sockets as shown.
2. Install bolts from outside of frame and secure with threaded knobs on inside of frame at desired height. Hand tighten.
3. Install cable ties through back upholstery grommets and attach around frame rail as shown.

- ⚠ **WARNING:** Ensure both backposts are the same height and all parts of Adjustable Back are secured as instructed before use.



1.770.368.4700

Information contained herein is subject to change.
The most current and complete product information can be found on our website.
www.grahamfield.com

Follow us on

© 2014, GF Health Products, Inc. All Rights Reserved.
E&J, Everest & Jennings, and Graham-Field are trademarks of GF Health Products, Inc., One Graham-Field Way, Atlanta GA 30340-3140
GF Health Products, Inc. is an ISO 13485 Certified Company.