



6497A: MULTI-POSITION OPEN PADDED RAISED TOILET SEAT INSTALLATION AND OPERATION INSTRUCTIONS

PLEASE SAVE THESE INSTRUCTIONS FOR FUTURE USE

Info: The most current version of these instructions can be found online at www.grahamfield.com.

- ⚠ **WARNING: Important! Read and understand these instructions before installing or using the raised toilet seat. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the raised toilet seat is not properly assembled and installed, personal injury and damage to the raised toilet seat could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the raised toilet seat.**
- ⚠ **WARNING: This product has a maximum weight capacity of 300 lb, EVENLY DISTRIBUTED.**
- ⚠ **WARNING: Notice for California Customers - California Proposition 65 WARNING: This product can expose you to chemicals including DEHP which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to www.P65Warnings.ca.gov/product.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**



6497A

INTENDED USE

The raised toilet seat is intended for use by those who have difficulty sitting down on or standing up from the toilet.

UNPACKING


Remove raised toilet seat, hardware, and protective packaging from carton. Remove the hardware bag from inside the seat and shield assembly.

ASSEMBLY

If the splashguard is not attached to the raised toilet seat, perform the following steps to attach it.

Tools required: Phillips or blade screwdriver

1. Remove hardware containing three screws from bag.
2. Align the three splashguard holes with the three holes on the bottom of the toilet seat, as shown at right.
3. Use a Phillips or blade screwdriver to screw the splashguard to the toilet seat with the provided screws. Start on one side and work your way around to align the holes and install the screws.

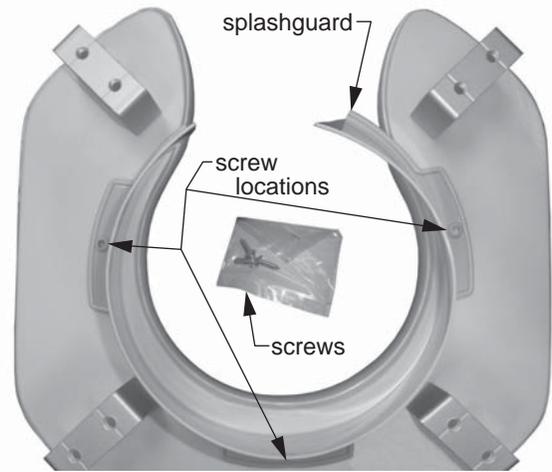

6497A: underside with splashguard

INSTALLATION

See picture at right with components labeled. The raised toilet seat has four rim clamps (two front and two rear) that attach it to the toilet rim, and four seat brackets (two front and two rear), into which the clamps screw, that adjust seat height.


1. Screw the two front rim clamps into the front seat brackets and adjust seat front to desired height. Ensure the height of both front clamps is the same.
2. Screw the two back rim clamps into the back seat brackets and adjust seat back to desired height. Ensure the height of both back clamps is the same.

⚠ WARNING: For stability, ensure front clamps are adjusted to the same height and back clamps are adjusted to the same height. For certain situations such as postoperative, hip conditions or arthritis, seat back can be adjusted slightly higher than seat front (front clamps don't need to be adjusted to the same height as back clamps).

3. Align the back rim clamps and front rim clamps to the toilet seat rim and press down on raised toilet seat to snap them on. If the clamps are too loose, squeeze them closed slightly to ensure a tight fit.

Locking Clamps

Tighten the wing nuts until the sides of the clamps are secure against the sides of the toilet bowl rim.

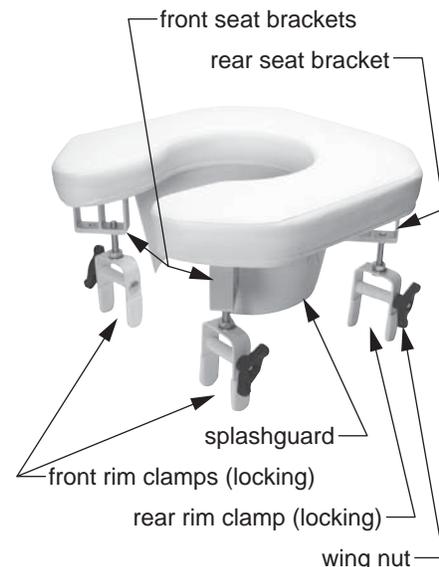

6497A

OPERATION

- ⚠ **WARNING:** Always ensure the raised toilet seat is securely installed and fits tightly on toilet bowl before use.
- ⚠ **WARNING:** Users with limited physical capabilities should be supervised or assisted when using raised toilet seat.

MAINTENANCE / CLEANING

- ⚠ **WARNING:** Check the raised toilet seat weekly for signs of damage or cracking. If damage is found, **DO NOT** use the raised toilet seat. Contact your Graham-Field authorized distributor immediately.

Loosen, retighten, and check for secure fit of raised toilet seat weekly. To clean raised toilet seat, wash with soap and water or a mild household detergent and rinse with clear water. Ensure the raised toilet seat is rinsed well before use.

- ▲ **NOTICE:** **DO NOT** use abrasive cleaners or scouring pads on any part of the raised toilet seat.

The padded toilet seat's heavy-duty vinyl material will provide excellent service under normal conditions. However, care must be taken to avoid puncturing or cutting the vinyl. If the seat is damaged, avoid further use for sanitary reasons. Contact your Graham-Field authorized distributor dealer to inquire about replacement seats. Do not submerge rear portion of padded toilet seat in water when cleaning. Keep water away from rear air-bleed hole located on underside of seat.

LIMITED WARRANTY

SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, this product is warranted for six (6) months. The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1-770-368-4700, sending a fax request to 1-770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.



1.770.368.4700

Information contained herein is subject to change.
The most current and complete product information can be found on our website.
www.grahamfield.com



© 2011, GF Health Products, Inc. All Rights Reserved.
Lumex and Graham-Field are trademarks of
GF Health Products, Inc., One Graham-Field Way, Atlanta GA 30340-3140
GF Health Products, Inc. is an ISO 13485 Certified Company.

Manufactured for GF Health Products, Inc.
Made in China