

LUMEX®

A Graham-Field Family Brand

Handheld Shower Holder

Universal Fit
2 Mounting Options:
Suction or Screw Installation



Installation:

- Remove shower holder from mounting bracket.
 - Clean and dry desired mounting area.
 - Place holder on desired location pointing upward.
 - Rotate holder clockwise $\frac{1}{4}$ turn to lock in place.
-
- Retira el soporte de la ducha del soporte de montaje.
 - Limpia y seca la superficie de montaje deseada.
 - Coloca el soporte en el lugar deseado, apuntando hacia arriba.
 - Rota el soporte $\frac{1}{4}$ de vuelta para asegurarlo en su lugar.

1 Year Warranty
1 Año de Garantía

Screw Installation:

- Clean and dry desired mounting area.
 - Use a pencil to mark screw location through mounting base.
 - Remove mounting base. Use a $\frac{1}{4}$ in. drill bit to gently drill hole at pencil marks.
 - Insert wall anchor. Align mounting base with hole. Assure "top" is oriented upwards. Install screws. **DO NOT OVERTIGHTEN.**
 - Slide shower holder pointing upward onto mounting base. Rotate holder clockwise.
-
- Limpia y seca la superficie de montaje deseada.
 - Marca la ubicación de los tornillos con un lápiz a través de la base de montaje.
 - Retira la base de montaje. Usa una broca de taladro de $\frac{1}{4}$ in. para abrir un orificio en la marca del tornillo.
 - Inserta el anclaje de pared. Alinea la base de montaje con el orificio. Asegúrate de que "top" (parte superior) apunte hacia arriba. Coloca el tornillo. **NO APRIETES DESMASIADO.**
 - Desliza el soporte de ducha en la base de montaje, apuntando hacia arriba. Rota el soporte en sentido de las manecillas del reloj para asegurarlo en su lugar.

Manufactured for:
GF Health Products, Inc.
2935 Northeast Parkway
Atlanta, Georgia 30360
Made in China
www.grahamfield.com



graham
field

LS
Latex
Statement

△ WARNING: Notice for California Customers - California Proposition 65
WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.

*This product contains natural rubber latex, which may cause allergic reactions