

*Info: The most current version of these instructions can be found online at [www.grahamfield.com](http://www.grahamfield.com).*

- ⚠ **WARNING: Important! Read and understand these instructions before installing or using the Toilet Seat Rail. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If this product is not properly assembled and installed, personal injury and damage could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage.**
- ⚠ **WARNING: This product has a maximum weight capacity of 250 lb, EVENLY DISTRIBUTED.**
- ⚠ **WARNING: Notice for California Customers - California Proposition 65 WARNING: This product can expose you to chemicals including DEHP which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to [www.P65Warnings.ca.gov/product](http://www.P65Warnings.ca.gov/product).**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

### INTENDED USE

The Toilet Seat Rail is intended to provide support and stability during toileting.

### UNPACKING

Remove Toilet Seat Rail from carton and protective packaging. Check for obvious damage to the carton or its contents. If damage is evident, do not use the product; please notify the carrier and your Graham-Field authorized distributor for further instruction.

### ASSEMBLY (REFER TO ILLUSTRATIONS)



1. Find the two components that are labeled with a (1) sticker. Align the stickers and push the two pieces together until the push buttons make an audible click.
2. Find the two components that are labeled with a (2) sticker. Align the stickers and push the two pieces together until the push buttons make an audible click.




3. Find the two components that are labeled with an **(A)** sticker. Align the stickers and push the two pieces together until the holes align.
4. Press the C-clip through the aligned holes until it is fully engaged.
5. Find the two components that are labeled with a **(B)** sticker. Align the stickers and push the two pieces together until the holes align.
6. Press the C-clip through the aligned holes until it is fully engaged.

## INSTALLATION (REFER TO ILLUSTRATIONS)

### Install Bracket

1. Remove existing toilet seat.

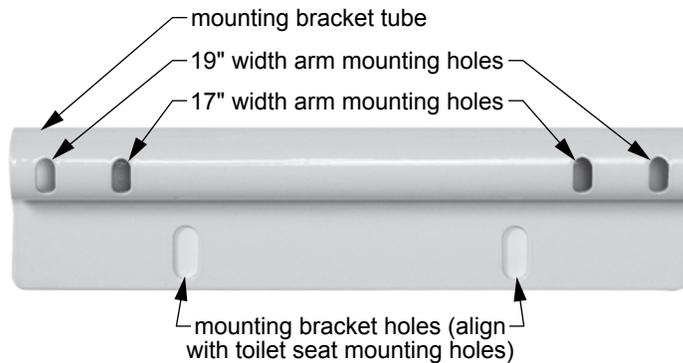

**6450R-2 mounting bracket**

2. Place mounting bracket on flat surface at rear of bowl and align mounting bracket holes with toilet seat mounting holes. Ensure that mounting bracket tube is to the rear, closest to the tank.
3. Replace toilet seat, inserting bolts through mounting bracket holes and into toilet seat mounting holes. Replace original nuts and tighten.

## Install Arms



1. Depress push buttons and install arms in mounting bracket as shown above. Ensure the Toilet Seat Rail sleeves rest securely on the toilet rim.
2. Adjust Toilet Seat Rail arm width by engaging push buttons in desired holes in mounting bracket tube. Adjust the arms to either 17" width (inner holes) or 19" width (outer holes).

**⚠ WARNING: Ensure push buttons are locked in place before use.**

### OPERATION

**⚠ WARNING: Always ensure this product is securely installed before use.**

**⚠ WARNING: Users who have limited physical capabilities must be assisted or supervised when using this product.**

### MAINTENANCE / CLEANING

**⚠ WARNING: Check the toilet seat rail weekly for signs of damage or cracking. If damage is found, DO NOT use the toilet seat rail. Contact your Lumex authorized distributor immediately.**

Loosen, retighten, and check for secure fit of toilet seat rail weekly. To clean toilet seat rail, wash with soap and water or a mild household detergent and rinse with clear water. Dry with a clean, soft cloth. Ensure the toilet seat rail is rinsed well and dried before use.

**▲ NOTICE: Never use abrasive cleaners or scouring pads on any part of the toilet seat rail.**

## LIMITED WARRANTY

### SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, this product is warranted for six (6) months. The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

### OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1-770-368-4700, sending a fax request to 1-770-368-2386, or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

### EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

### ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

### NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.



1.770.368.4700

Information contained herein is subject to change.

The most current and complete product information can be found on our website.

[www.grahamfield.com](http://www.grahamfield.com)



© 2011, GF Health Products, Inc. All Rights Reserved.  
Lumex and Graham-Field are trademarks of GF Health Products, Inc.  
GF Health Products, Inc. is an ISO 13485 Certified Company.

Manufactured for GF Health Products, Inc.  
Made in China