


SAVE THESE INSTRUCTIONS FOR FUTURE USE

Tools required are listed at the beginning of each section


- ⚠ **WARNING: Important! Read and understand these instructions before installing or using the O₂ / IV Holder. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the O₂ / IV Holder is not properly assembled and installed, personal injury and damage to the O₂ / IV Holder could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Everest & Jennings replacement parts. Non-Everest & Jennings replacement parts could cause personal injury and damage to the O₂ / IV Holder.**
- ⚠ **WARNING: Cancer and Reproductive Harm - www.p65warnings.ca.gov.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

E&J™ ADVANTAGE®, ADVANTAGE LX, TRAVELER®, TRAVELER HD, TRAVELER HTC, TRAVELER L3, TRAVELER L3 PLUS, TRAVELER L4, TRAVELER SE, AND TRAVELER SE PLUS: HOLDER INSTALLATION ON WHEELCHAIR (SEE ILLUSTRATIONS BELOW)

Remove O₂ / IV Holder from carton and protective packaging.

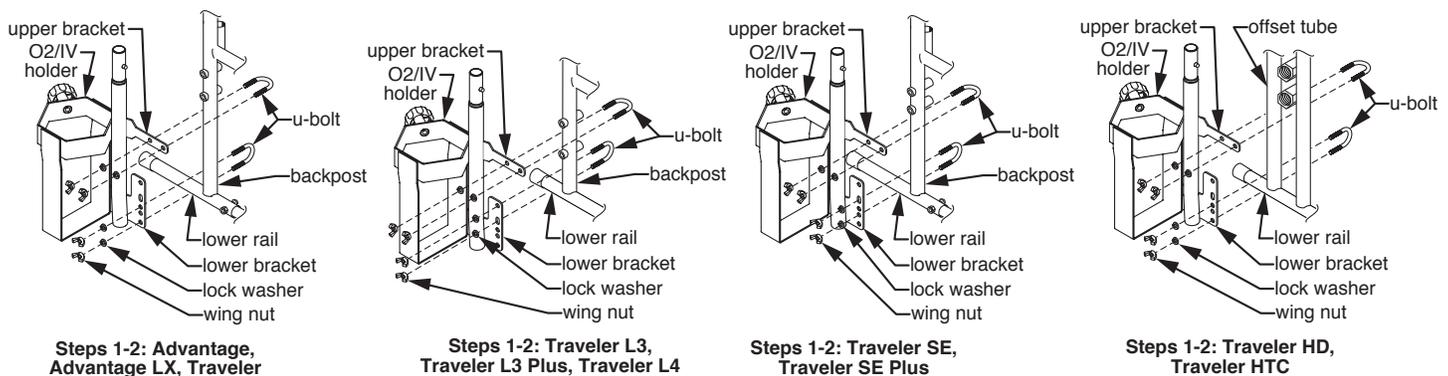

1. Note that left sideframe above is shown from inside. Install lower holder section on left-rear sideframe lower rail: Insert U-bolt over rail, then through appropriate lower holder bracket holes as shown above. Install washers and wing nuts on U-bolt ends, securing holder to rail.
2. Install on sideframe backpost: Insert U-bolt over backpost (except Traveler HD: insert U-bolt over offset tube behind backpost, as shown above), then through upper bracket as shown above. Install washers and wing nuts on U-bolt ends, securing holder to backpost/offset tube.
3. Tighten all four wing nuts until holder is secure on sideframe as shown at right. Ensure holder is securely attached before proceeding to HOLDER ASSEMBLY.

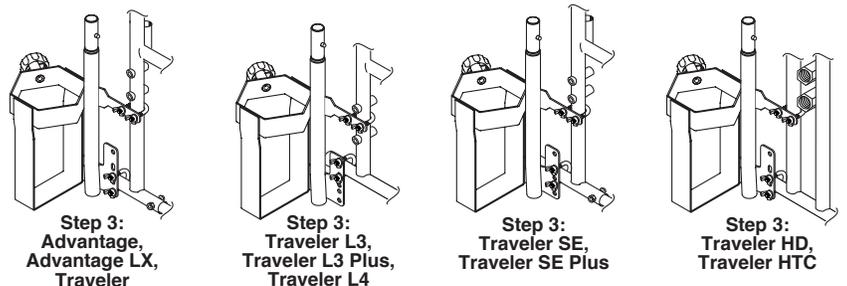


PARAMOUNT™ XD: HOLDER AND ADAPTER INSTALLATION ON WHEELCHAIR (SEE ILLUSTRATIONS AT RIGHT)

Tools required: Phillips screwdriver, M5 hex key, M13 wrench


Remove O₂ / IV Holder and adapter from carton and protective packaging.

1. Use M5 hex key and M13 wrench to remove the two socket head screws in left sideframe backpost, located directly above the rear wheel axle block, and replace them with the M8 x 45mm hex socket head screws, as shown in Step 1 illustration at right. Note that left sideframe is shown from inside.

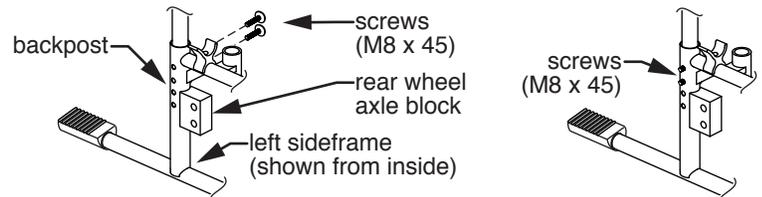


Step 1: Install M8 x 45 screws

2. Assemble T-plate to the upper holder bracket with the two M8 flat head screws and lock nuts. **The T-plate must be located between the sideframe and upper holder bracket.** Tighten the lock nuts securely.

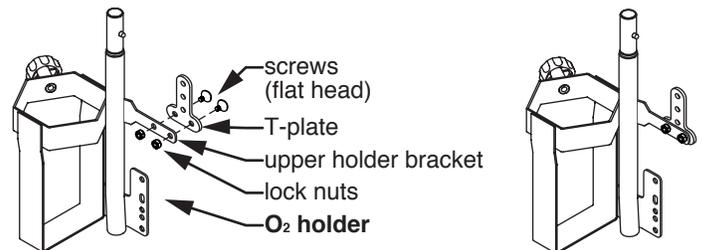


Step 2: Assemble T-plate to upper holder bracket

3. Install the upper holder bracket with T-plate to the inside of the left frame by aligning the T-plate with the M8 x 45mm hex socket head screw ends and installing the M8 lock nuts. Tighten the lock nuts securely.

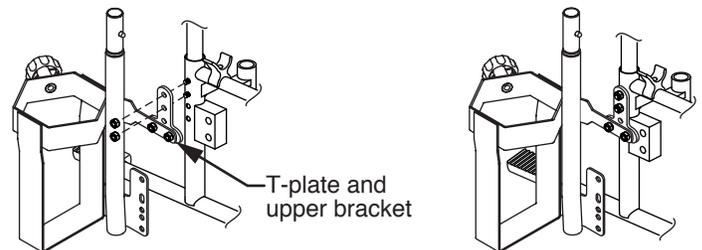

Step 3: Install T-plate on inside of frame

4. Place the short spacer plate between the sideframe lower rail and the lower holder bracket and assemble the U-bolt, lock washers, and wing nuts. Tighten the wing nuts securely.
5. Ensure holder is securely attached before proceeding to HOLDER ASSEMBLY.
6. The remaining U-bolt, washers, and wing nuts may be discarded.

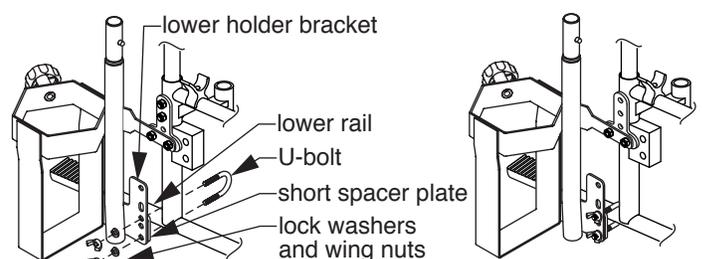

Step 4: Assemble lower holder bracket to lower rail

HOLDER ASSEMBLY

1. Install holder middle section: slide middle tube down over lower tube until button lock locks into hole as shown at right.

⚠ WARNING: Ensure button lock is locked in place before continuing.

2. Install holder upper section (IV hanger): Loosen height adjustment knob as necessary. Slide upper tube down into middle tube until desired height is reached, then tighten height-adjustment knob until secure.

Adjust IV hanger height


3. Loosen knob until hanger slides freely inside tube.
4. Slide hanger to desired height. Tighten knob until hanger is secure in tube.

OPERATION


⚠ WARNING: Ensure the O₂ / IV Holder is installed as described, and all components are securely fastened, before use.

Install O₂ cylinder

Loosen anti-rattle knob as necessary, install cylinder, and tighten knob enough to prevent cylinder from rattling.

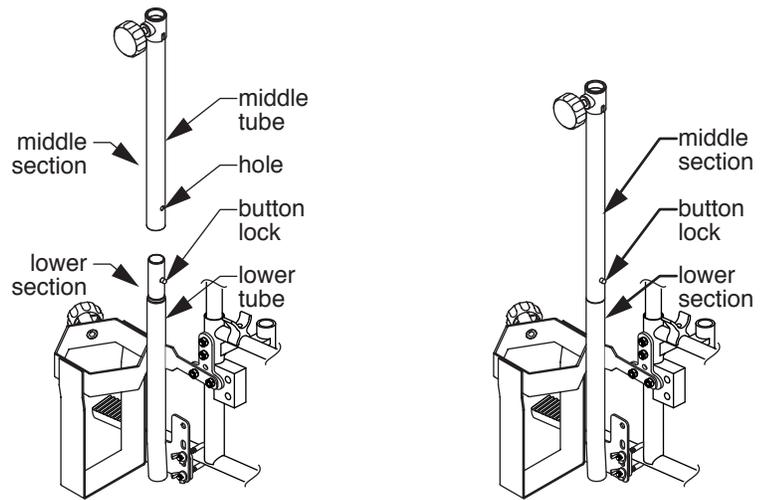

Step 1: Install middle section

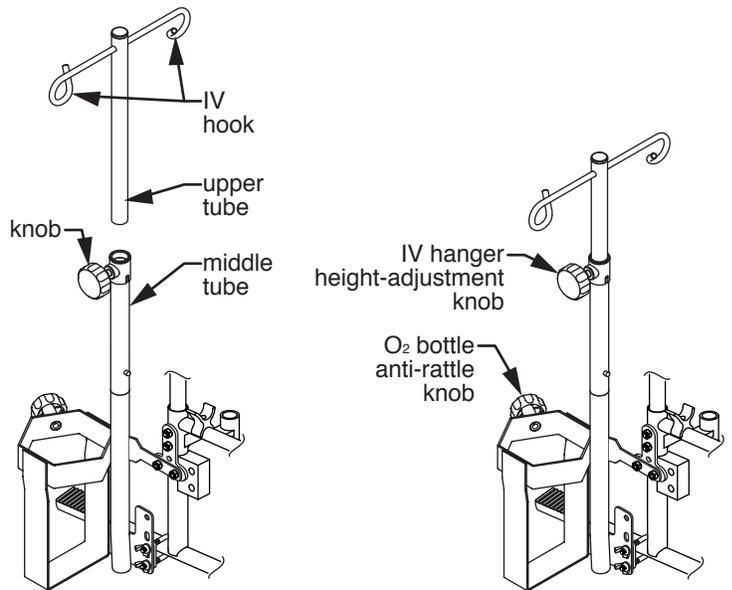


Step 2: Install IV hanger

ALL MODELS: LOCK MIDDLE TUBE IN PLACE WITH BOLT (OPTIONAL)

Tools required: Needlenose pliers, Phillips screwdriver, 7/16" wrench

You may wish to use the included optional bolt and nut to more securely attach the middle tube to the lower section.

1. Depress button lock and remove middle tube from lower tube.
2. Remove button lock from lower tube: Depress button while holding button lock with needlenose pliers. Remove button lock from holder as shown at near right.
3. Slide middle tube down over lower tube until holes match. Insert bolt through both tubes as shown at far right.
4. Install nut on bolt. Use Phillips screwdriver and 7/16" wrench to tighten until both tubes are secure.

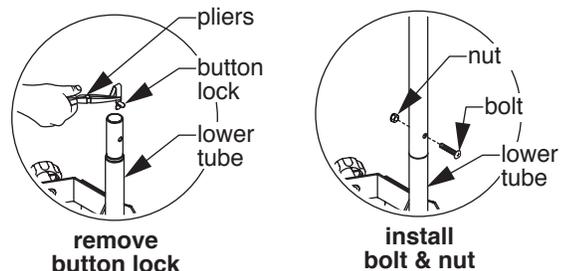

MAINTENANCE

⚠ WARNING: Inspect the O₂ / IV Holder frequently and tighten it as needed to ensure secure installation.

Check holder at least once a week to ensure all parts lock securely into place and will not accidentally unlock. Wipe holder with a soft cloth and dry immediately if exposed to moisture. If necessary, wash with a mild soap and water solution, rinse, and dry with a soft cloth.

▲ NOTICE: Never use abrasive cleaners or scouring pads on any part of the O₂ / IV Holder.

LIMITED WARRANTY

SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, this product is warranted for three (3) months. The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1.770.368.4700, sending a fax request to 1.770.368.2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.



1.770.368.4700

Information contained herein is subject to change.

The most current and complete product information can be found on our website.

www.grahamfield.com

Follow us on



© 2009, GF Health Products, Inc. All Rights Reserved.

Advantage, E&J, Everest & Jennings, Paramount, and Traveler are trademarks of GF Health Products, Inc., One Graham-Field Way, Atlanta GA 30340-3140

GF Health Products, Inc. is an ISO 13485:2016 /MDSAP Certified Company. • Manufactured for GF Health Products, Inc. • Made in Taiwan