

How To Replace Your Nebulizer Filter



Replacing the filter on your nebulizer is one of the most important aspects of nebulizer maintenance. It is just as important as replacing your neb accessory set, since without the filtered air coming from the compressor, you cannot take a nebulizer treatment. Replacing your nebulizer's filter is easy, and should be performed regularly.

1. Wash and dry hands thoroughly.



2. Place your nebulizer on a flat surface and make sure that the ON/OFF switch is in the OFF position.



3. Remove the filter cap located next to the ON/OFF switch, sometimes a coin may be needed.



4. Remove the dirty filter with your hand and throw away-DO NOT wash it!



5. Insert new filter and reattach filter cap.
You are now ready to use your nebulizer for treatment.



A good way to remember to change out your nebulizer filter is to check it before each treatment. If the filter is grey, it is time for a new one. By doing this you will always have a good filter when taking your treatments, reducing the possibility of an infection.

For more information on the John Bunn line of nebulizers, visit our website at www.grahamfield.com



GF1500057RevA15
© 2015 GF Health Products, Inc.
Atlanta, GA 30360

www.grahamfield.com