

# LUMISCOPE®

*For The Quality Of Life™*



## Single Head Stethoscope

**Latex®  
FREE**

Model 006

The Lumiscope Single Head Stethoscope is indicated for use in the detection and amplification of sounds associated with various organs of the human body.

### Parts Identification:



1	Ear Tips
2	Binaural
3	Spring
4	Single Head Chestpiece
5	Y-Tubing

### Assembly:

Connect the binaural and the single head chestpiece to the Y-tubing.

## Usage:

- Gently place an ear tip in each ear.
- Place the chestpiece on the appropriate body part. Move the chestpiece around slowly to obtain maximum amplification and clarity.

## Caution:

- Use only under direct adult supervision.
- Do not use this device for any purpose other than the intended use indicated.
- Do not use this device if any of the components are missing or damaged. Contact your place of purchase for return or replacement information.
- Never use this device without ear tips in the proper place.
- Avoid exposure of this device to high temperatures.
- Clean ear tips with a soft pad or cotton swabs moistened with alcohol. Do not use any other chemicals.

**Store device in a cool, dry environment**

Manufactured for:  
GF Health Products, Inc.  
Atlanta, GA 30360  
Made in China  
[www.grahamfield.com](http://www.grahamfield.com)  
Repair Department: 1-800-672-8293