


OPTION ACCESSORIES



**Model# 7958A
with options 7952A and 7953**

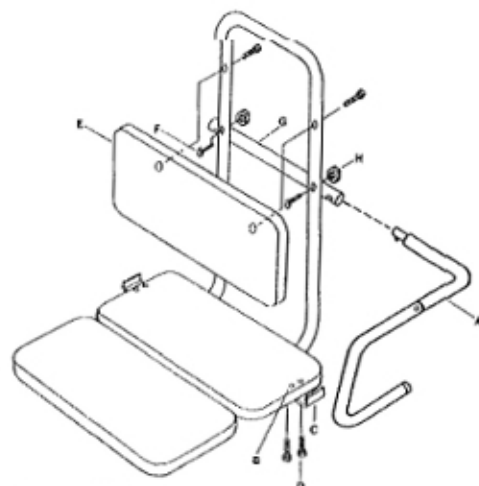

7952A Pivoting Support Arm

- 1) Remove seat frame assembly from unit. Place seat frame assembly upside down on a work surface.
- 2) Determine on which side the pivoting support arm (A) is to be used. This is determined by the location of the faucets. The pivoting support arm should be located on the side next to the transfer pad.
- 3) Locate the two screw holes (B) in the rear seat board. They are approximately 1" in from the side; one is 3" from the back, and the other is 4 - 7/8" from the back. Pierce the vinyl at these two locations.
- 4) Mount arm stop bracket (C) with (2) two short Phillips head screws (D) so that the kick bend faces towards the top of the seat.
- 5) Remove the back rest pad (E) from the unit.
- 6) Place the two (2) curved head screws (F) through the back bend so they face out.
- 7) Place the straight tube (G) over the screws so the slot is facing down. Add the nuts (H) to the screws and tighten.
- 8) Replace back rest pad.
- 9) Place seat assembly onto frame. Add swing arm to the unit by depressing the push button and engaging the arm into the back tube until the push button engages into the slot.

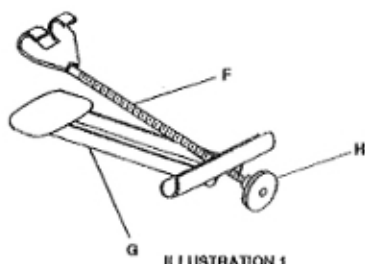


ILLUSTRATION 1

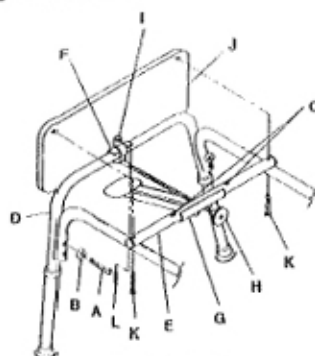


ILLUSTRATION 2

7953 Bathtub Safety Clamp

- 1) Before attaching the tub clamp to the transfer bench, assemble tub clamp by unscrewing adjustment knob (H) inserting "Y"-hook rod (F) through top hole of the clamp arm (G) and re-attach adjustment knob. (SEE ILLUSTRATION 1)
- 2) If transfer bench has already been assembled, unclamp back and seat from frame. (see transfer bench instruction sheet.)

REFER TO ILLUSTRATION 2 FOR THE FOLLOWING INSTRUCTIONS

- 3) Remove and save screws (K) which secure the transfer pad (J). Loosen small clip screw (L). DO NOT REMOVE Rotate the transfer pad (J) 90 degrees as shown.
- 4) Remove top screw (A). Insert rubber stop (C) to the top screw (A) as shown. Tighten. Repeat for the other side (not shown).
- 5) Install the two small locator screws (D) into the holes located on the top of the support tube (E).
- 6) Place clamp assembly (G) on the top of support tube (E) between the two small locator screws (D). Let threaded "Y"-hook hang down under side frame (D).
- 7) Rotate transfer pad (J) on the frame, align holes and re-attach screws (K), do not tighten.
- 8) Position threaded "Y"-hook (F) around slat clip (I) and push to snap into place. Tighten transfer pad screws (K) and small clip screw (L).
- 9) Place unit into tub with the rubber pad of the clamp assembly (G) to the inside of tub. Adjust the height of the unit so it is at the lowest possible position without hitting the crown of the tub wall (see transfer bench instructions on how to adjust bench height).
- 10) Tighten the clamp arm by turning knob (H) in a clockwise direction.
- 11) Once frame is tightly installed in tub, reattach back and seat assembly by snapping into place on the frame (see transfer bench instructions).

LUMEX®
for the quality of life

7952A Pivoting Support Arm

CALL OUTS

- A - Pivoting Support Arm
- B - Screw Holes (2)
- C - Arm Stop Bracket
- D - Short Phillips-head Screws (2)
- E - Backrest Pad
- F - Curved Head Screws (2)
- G - Straight Tube
- H - Nuts (2)

7953 Bathtub Safety Clamp

CALL OUTS

- A - Top Screw
- B - Rubber Stop
- C - Small Sheet Metal Screws
- D - Side Frame
- E - "Y"-Hook Rod
- G - Clamp Assembly
- H - Knob
- I - Slat Clip
- J - Seat Slat
- K - Screws (2)
- L - Small Clip Screw