
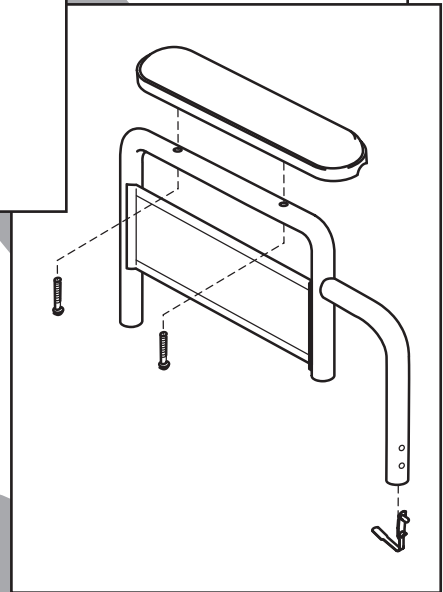
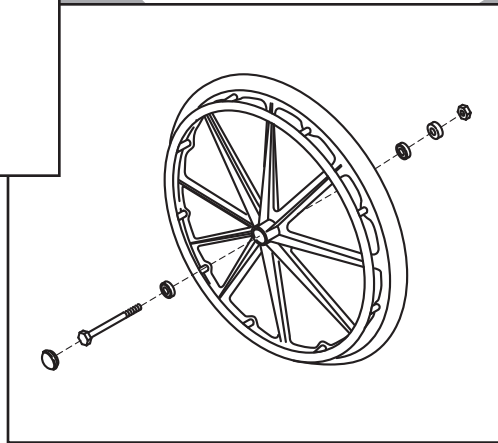


Traveler[®] L3

PARTS CATALOG

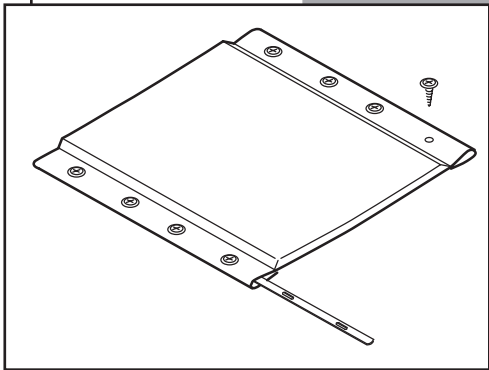

CONTENTS

SIDEFAME AND CROSSBRACE 3

ARM, DETACHABLE..... 4

UPHOLSTERY 4

REAR WHEEL 5

CASTER..... 5

FRONT RIGGING, DETACHABLE 6

ACCESSORIES 7

TO ORDER

Tel: 770-368-4700 • Fax: 770-368-2386


Returns: Parts may not be returned without prior authorization from GF Health Products, Inc. Custom parts cannot be returned for credit. Please refer to GF Health Products, Inc. Terms and Conditions for further information.

GF Health Products, Inc. is not responsible for typographical errors. All illustrations, specifications, packaging and warranties contained in this catalog are based on the latest product information available at the time of printing. The most current product information, including the most current revision of this catalog, can be found online at www.grahamfield.com.

Everest & Jennings, E&J, Traveler, and Graham-Field are registered trademarks of GF Health Products, Inc.

© 2009, GF Health Products, Inc., All Rights Reserved

SIDEFRAME AND CROSSBRACE

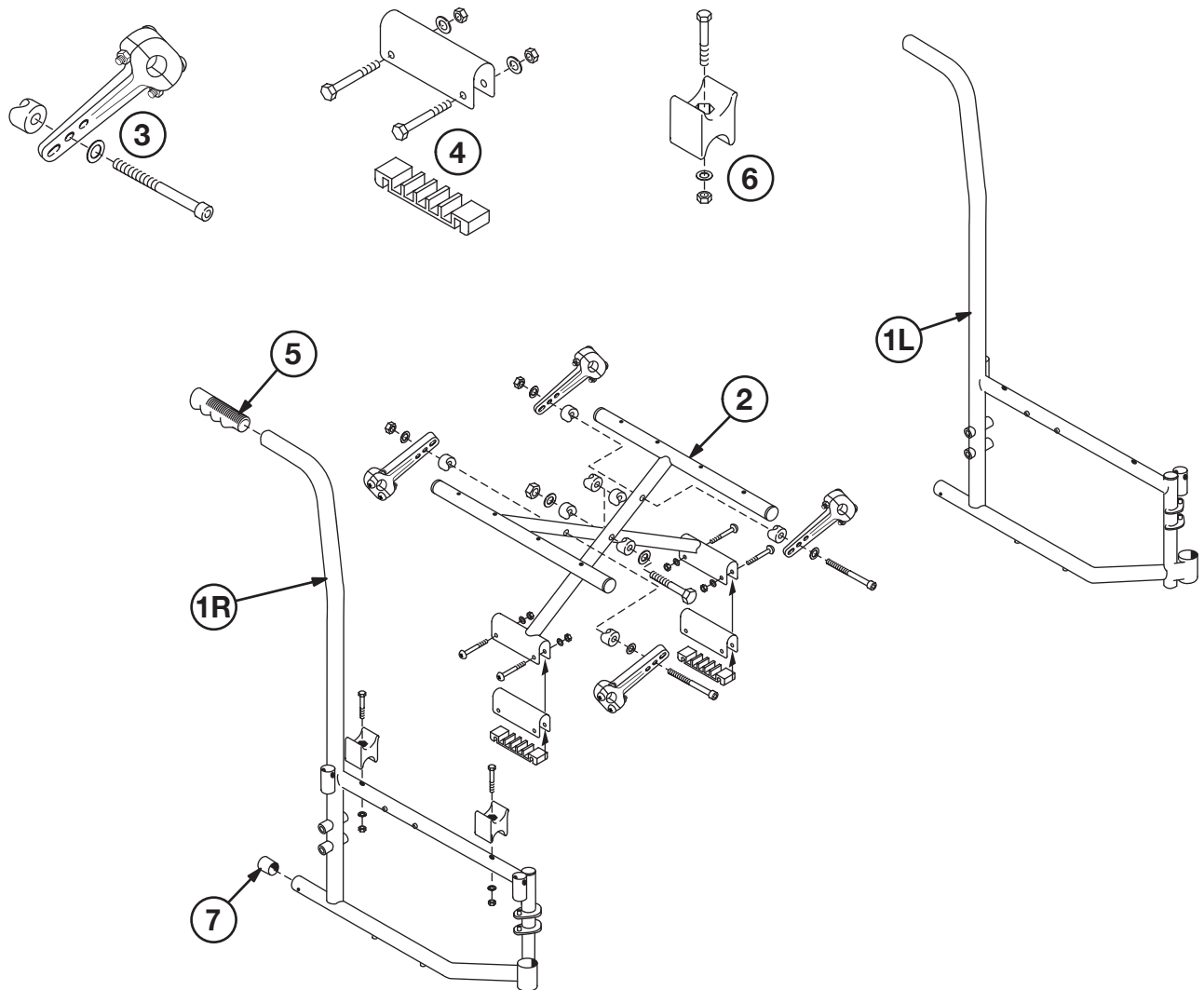

1 Sideframe, all widths

Right 90763301
Left 90763302

2 Crossbrace, pair (Includes center bolt and nut)

16" W 907632C1
18" W 907632C2
20" W 907632C3

3 Crossbrace link, set of 4 (includes mounting hardware)

16" W and 18" W 907632D1
20" W 907632D3


4 Crossbrace saddle bushing and gasket, pair (Includes mounting hardware) 90763205

5 Grip (10 each) 90763005

6 Seat rail cradle, pair - 2 pairs required per chair (includes 2 each screw, washer and nut)..... 90763206


7 Tipping sleeve (10 each)..... 90763006

ARM, DETACHABLE

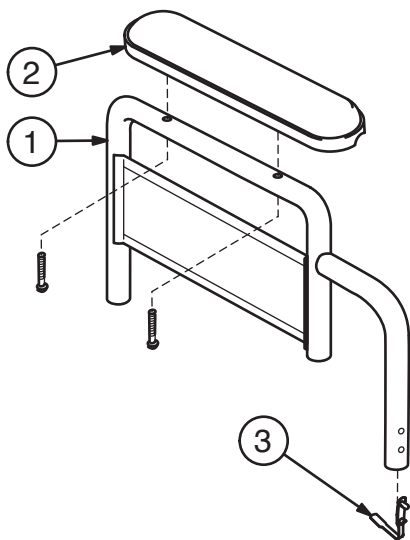

- 1 Arm, detachable fixed-height, pair (includes right and left arm with mounting hardware and side panel; does not include arm pad)
 - Desk length..... 90763305
 - Full length..... 90763306
- 2 Arm pad, pair (includes two black arm pads and mounting screws)
 - Desk length..... 90763022
 - Full length..... 90763023
- 3 Arm latch, pair, with hardware 90763223


UPHOLSTERY

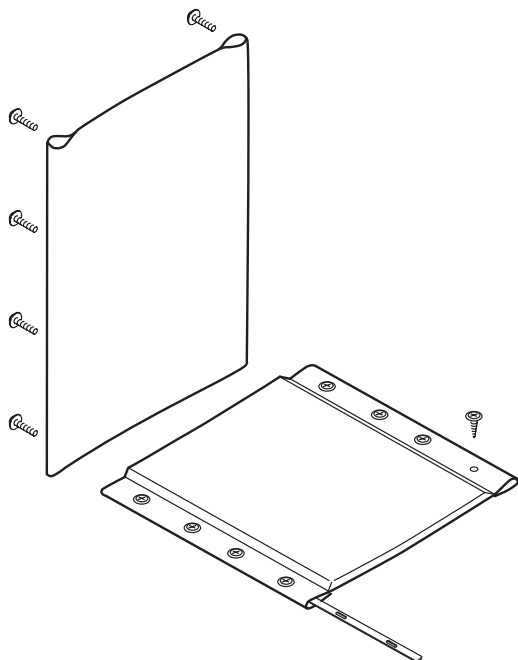

- Back upholstery, black nylon, and mounting hardware
- 16" W..... 907633A1
 - 18" W..... 907633A2
 - 20" W..... 907633A3


- Seat upholstery, black nylon, and mounting hardware
- 16" W x 16" D..... 907632B1
 - 18" W x 16" D..... 907632B2
 - 20" W x 16" D..... 907632B3

REAR WHEEL

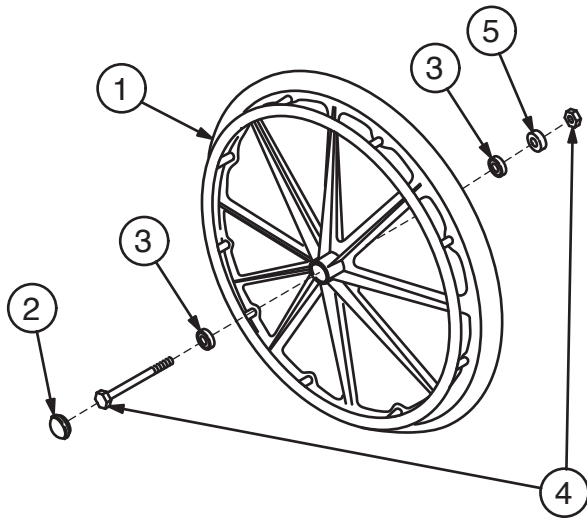


- | | |
|---|----------|
| 1 Rear wheel with handrim, 24" x 1", polyurethane tire (includes hub cap and bearing; does not include axle or nut) | 90763150 |
| 2 Hub cap (10 each)..... | 90763051 |
| 3 Wheel bearing (10 each) | 90763052 |
| 4 Rear axle and nut (10 each)..... | 90763054 |
| 5 Spacer (10 each)..... | 90763055 |

CASTER

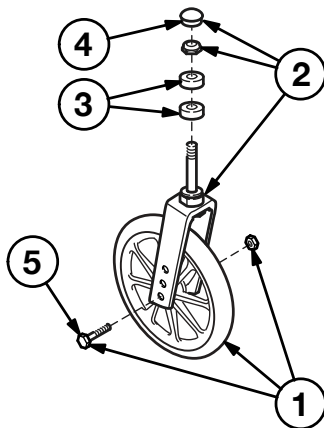


- | | |
|--|----------|
| Complete caster (fork and wheel), 8" x 1" solid rubber tire..... | 90763040 |
| 1 Caster wheel, axle and nut | 90763041 |
| 2 Caster fork, stem nut and dust cap | 90763042 |
| 3 Caster stem bearing (10 each) | 90763043 |
| 4 Caster dust cap (10 each)..... | 90763044 |
| 5 Caster axle and nut (10 each)..... | 90763045 |

FRONT RIGGING, DETACHABLE

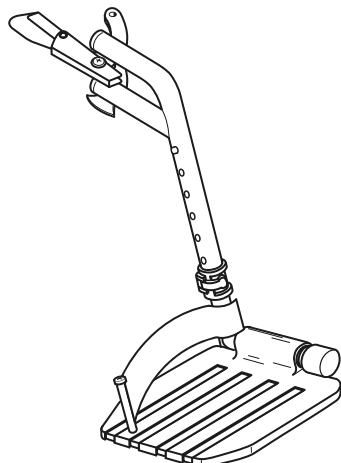


Footrest, pair, all widths
Includes right and left hanger with release
and composite footplate with extension
and heel loop 90763030

Composite footplate, pair
(Fits all front rigging) 90763033

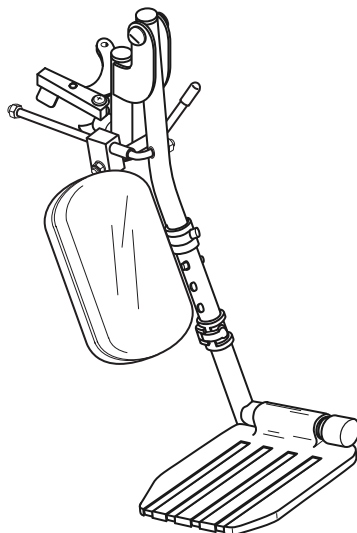

Low-pivot elevating legrest, pair, all widths
Includes right and left hanger with release,
composite footplate with extension, and
legrest panel (does not include
heel loop) 90763031

Elevating legrest panel kit
(Fits all elevating legrests)
Includes 2 black padded panels
and hardware 90763032

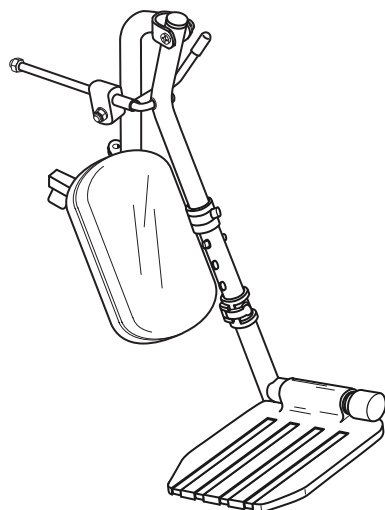

High-pivot elevating legrest, pair, all widths
Includes right and left hanger with release,
aluminum footplate with extension, and
legrest panel (does not include
heel loop) 90763435

Traveler L3 accessories (not shown)

1	Heel loop, pair.....	90763064
2	Wheel lock, pair (includes rubber tips)	
	Push to lock.....	90763060
	Pull to lock.....	90763061
3	Rubber tip, wheel lock (10 each).....	90763063
4	Wheel lock extension, pair	90763062
5	Positioning belt (metal buckle)	92133040
6	Anti-Fold bar	92523850
7	O2-IV holder	B10320F
8	O2 cylinder holder.....	B10325
9	Anti-Tippers, pair.....	90763230

Everest & Jennings®

**GF Health Products, Inc.
2935 Northeast Parkway
Atlanta, Georgia 30360 USA**

tel: 770-368-4700

fax: 770-368-2386



GRAHAM-FIELD.

www.grahamfield.com

**Traveler L3 Parts Catalog
GF0900104RevC14**