

Info: The most current version of these instructions can be found online at www.grahamfield.com.

- ⚠ **WARNING: Important! Read and understand these instructions before installing or using the Versaframe Toilet Safety Frame. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the Versaframe is not properly assembled and installed, personal injury and damage to the Versaframe could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the Versaframe.**
- ⚠ **WARNING: This product has a weight limit of 250 lb, EVENLY DISTRIBUTED.**
- ⚠ **WARNING: Users who have limited physical capabilities must be assisted or supervised when using the Versaframe.**
- ⚠ **WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

INSTALLATION


Unpacking

Remove Versaframe from carton and protective packaging. Check for obvious damage to the carton or its contents. If damage is evident, do not use the product; please notify the carrier and your Graham-Field authorized distributor for further instruction.

Installation

1. Remove existing toilet seat.
2. Place mounting bracket on flat surface of bowl and align holes. Ensure the tube on the bracket is to the rear, closest to the tank.
3. Replace toilet seat, inserting bolts through mounting bracket holes and into toilet seat mounting holes. Replace original nuts and tighten.
4. Depress push buttons and install arms in mounting bracket as shown.
5. Adjust width of Versaframe arms by engaging push buttons in desired holes in mounting bracket tube.

- ⚠ **WARNING: Ensure mounting bracket push buttons are locked in place before use.**

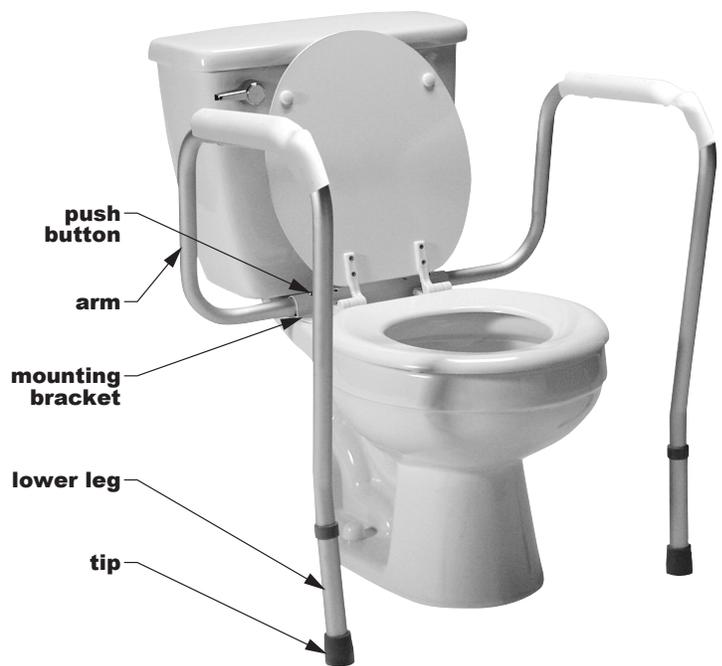

Versaframe toilet safety frame

Height Adjustment

1. Adjust height by depressing the lower leg height adjustment button, twisting the lower leg, and moving leg to desired height.
2. When the desired height is reached, twist the lower leg until you hear the height adjustment button click into place.

⚠ WARNING: Ensure that height adjustment push buttons are locked in place, and both legs are adjusted to the same height, before use.

⚠ WARNING: Ensure the Versaframe is installed as described. If the Versaframe is not properly assembled and installed, personal injury and damage to the Versaframe could result.

OPERATION

⚠ WARNING: Always ensure the Versaframe is securely installed before use.

MAINTENANCE

⚠ WARNING: Inspect the Versaframe weekly and tighten it as needed to ensure secure installation. Check all rubber tips for tears, rips or wear. If any of these conditions exist, replace tips IMMEDIATELY.

▲ NOTICE: Never use abrasive cleaners or scouring pads on any part of the Versaframe.

To clean the Versaframe, wash it with soap and water or a mild detergent, then rinse it with water.

LIMITED SIX-MONTH WARRANTY

GF Health Products, Inc. (“GF”) warrants to the original retail purchaser or lessee (either is the “Customer”) only, that it will replace or repair components, at GF’s sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the Lumex 6460A / 6460R / 6465A-1 Versaframe Toilet Safety Frame is warranted for six months. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component, in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must contact GF directly by calling 678-291-3207, sending a fax request to 770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. For the complete warranty for this product, please visit our website at www.grahamfield.com.



GRAHAM-FIELD
www.grahamfield.com

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, including the most current version of these instructions, please visit our website at www.grahamfield.com. **Graham-Field** and **Lumex** are registered trademarks of GF Health Products, Inc.

© 2009 GF Health Products, Inc. • 2935 Northeast Parkway, Atlanta, GA 30360 • tel 770-368-4700 • fax 770-368-2386 • Made in China