



188 ARCH EASER



INSTRUCTIONS FOR USE

CAUTION: This product contains natural rubber latex which may cause allergic reactions.

Read these instructions before using the Pipeer Arch Easer. Save these instructions for future reference.

The most current version of these instructions can be found online at www.grahamfield.com.

SAFETY

- ⚠ WARNING: DO NOT** use this product without proper instruction from a healthcare professional.
- ⚠ WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**

INTENDED USE


The intended use of the Arch Easer is the treatment of arch and foot-related problems. The Arch Easer is designed to be worn all day, as comfort permits, over or under a sock.

APPLICATION

1. Place Arch Easer wrap on the floor with fuzzy side down and the fastener strip up and to the outside of the foot, as shown at right.
2. Pull the Arch Easer around the foot, positioned around arch, as shown at right. Arch Easer should be snug, but not too tight.
3. Fasten hook and loop strips so that the edge of the Arch Easer material is in a line between the third and fourth toes, as shown at right.



 **WARNING: Do not wear Arch Easer while sleeping.**

 **WARNING: If any pain, numbness, tingling, or discoloration occurs, remove Arch Easer immediately until normal feeling and color return. People who have numbness in their feet should use Arch Easer with caution.**

CLEANING AND MAINTENANCE

Hand wash Arch Easer gently, using mild soap. Air dry.

▲ NOTICE: Do not use detergent or bleach to clean Arch Easer. Do not iron Arch Easer.

THIRTY-DAY LIMITED WARRANTY

The following warranty information is a SUMMARY of the complete Product Warranty provided by GF Health Products, Inc. For the complete warranty, please visit our website at www.grahamfield.com. GF Health Products, Inc. ("GF") warrants to the original retail purchaser or lessee (either is the "Customer") only, that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. The warranty is for parts only and does not include labor or costs of shipping. All warranties are conditioned upon the proper use of the product strictly in accordance with applicable GF instructions. Product provided to users as rentals must be serviced by the provider and inspected prior to delivery. Each user must be properly trained on the operation and safety of this product prior to use. Within the guidelines set forth in this document, the Pipeer 188 Arch Easer is warranted for thirty days. The applicable warranty period shall commence from date of shipment to the Customer, unless there is an expiration date on the component, in which case the warranty shall expire on the earlier of the warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Dealer from whom the Customer purchased the product within the applicable warranty period. If there is not a Dealer, you must

contact GF directly by calling 678-291-3207, sending a fax request to 770-368-2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following: 1) Defects, damage, or other conditions caused by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner; 2) Products not installed, used, or properly cleaned and maintained; and 3) Labor or shipping charges. Other exclusions apply.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

For the complete warranty for this product, please visit our website at www.grahamfield.com.



Manufactured by:
GF Health Products, Inc.
Atlanta, Georgia 30360
Proudly made in the U.S.A.

GF Health Products, Inc.
2935 Northeast Parkway
Atlanta, GA 30360
telephone: 770-368-4700
fax: 770-368-2386
www.grahamfield.com

© 2012 GF Health Products, Inc.

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, please visit our website at www.grahamfield.com. **Graham-Field** and **Pipeer** are registered trademarks of GF Health Products, Inc.