
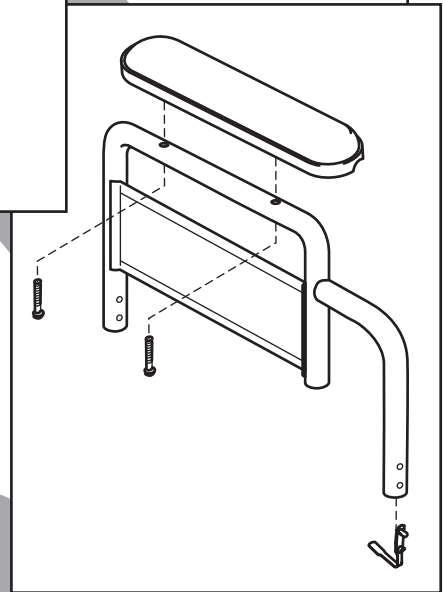
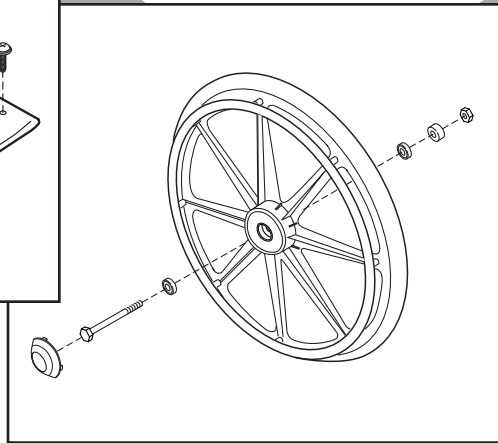


Traveler[®] LX

PARTS CATALOG

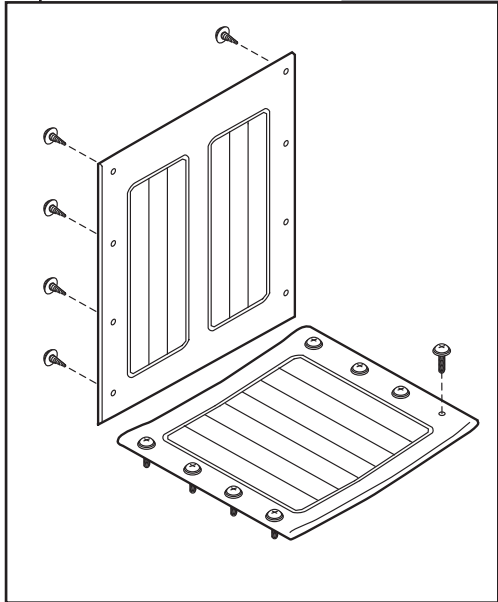

CONTENTS

SIDEFRAME (FIXED ARM), CROSSBRACE, ARM PAD KIT..... 3

SIDEFRAME (DETACHABLE ARM), CROSSBRACE 4

ARM, DETACHABLE 5

CASTER..... 5

UPHOLSTERY (ALL SIDEFRAMES)..... 5

FRONT RIGGING, DETACHABLE 6

REAR WHEEL..... 7

ACCESSORIES..... 7

TO ORDER

Tel: 770-368-4700 • Fax: 770-368-2386


Returns: Parts may not be returned without prior authorization from GF Health Products, Inc. Custom parts cannot be returned for credit. Please refer to GF Health Products, Inc. Terms and Conditions for further information.

GF Health Products, Inc. is not responsible for typographical errors. All illustrations, specifications, packaging and warranties contained in this catalog are based on the latest product information available at the time of printing. The most current product information, including the most current revision of this catalog, can be found online at www.grahamfield.com.

Everest & Jennings, E&J, Traveler, and Graham-Field are registered trademarks of GF Health Products, Inc.

© 2014, GF Health Products, Inc., All Rights Reserved

SIDEFRADE (FIXED ARM), CROSSBRACE, ARM PAD KIT

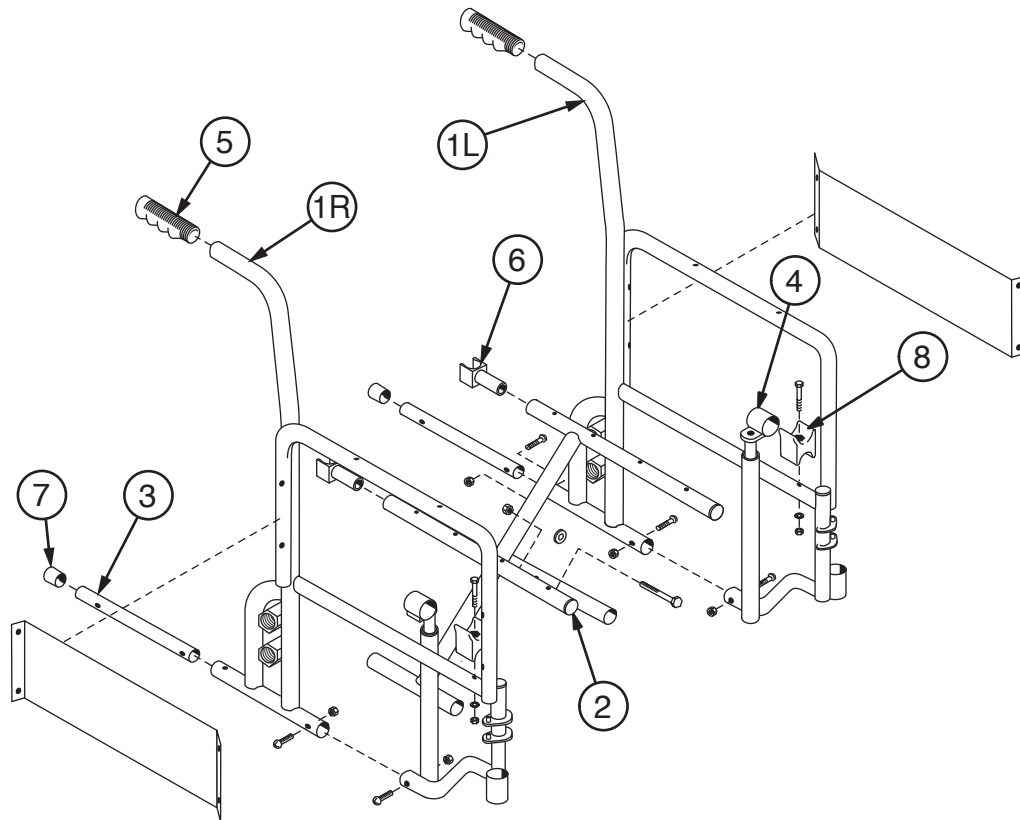

1 Sideframe, fixed arm, all widths (includes side panel)

Right 90763101
Left 90763102

2 Crossbrace, pair (includes center bolt and nut)

16" W 907631C1
18" W 907631C2
20" W 907631C3

3 Bottom inner rail kit (includes bottom inner rail (2 each) and screw and nut that hold bottom inner rail (4 each))

..... 90763003


4 Seat slide (2 each) 90763013

5 Grip (10 each) 90763005

6 Seat rail guide, rear (2 each) 90763015


7 Tipping sleeve (10 each) 90763006

8 Seat rail, cradle, screw, washer and
nut, front (2 each)
..... 90763403

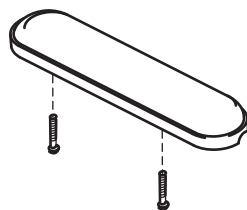

Arm pad kit
Includes 2 blue full length
arm pads and mounting screws 90763123

SIDEFRADE (DETACHABLE ARM), CROSSBRACE

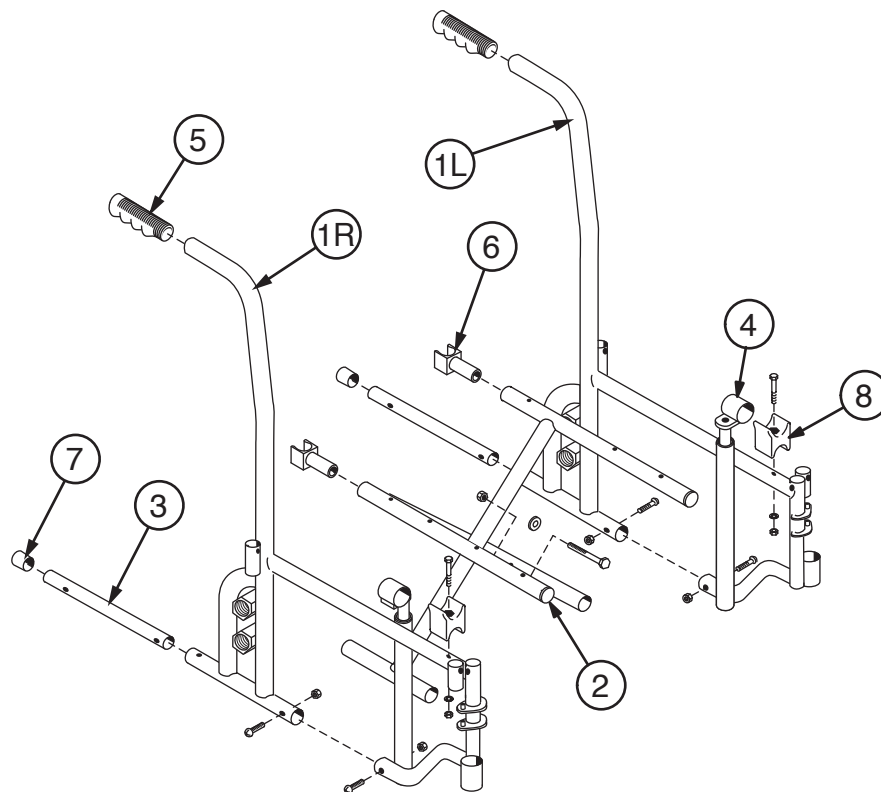

1 Sideframe, detachable arm, all widths

Right 90763111
Left 90763112

2 Crossbrace, pair (Includes center bolt and nut)

16" W 907631C1
18" W 907631C2
20" W 907631C3

3 Bottom inner rail kit (includes bottom inner rail (2 each) and screw and nut that hold bottom inner rail (4 each))

..... 90763003

4 Seat slide (2 each) 90763013


5 Grip (10 each) 90763005

6 Seat rail guide, rear (2 each) 90763015

7 Tipping sleeve (10 each) 90763006


8 Seat rail, cradle, screw, washer and nut, front (2 each)
..... 90763403

ARM, DETACHABLE

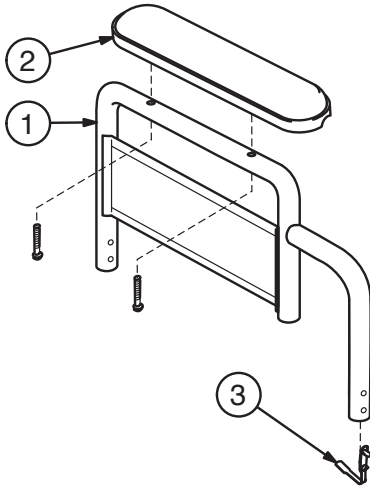

- 1 Arm, detachable fixed-height, pair (includes right and left arm with side panel, arm pad, and spring lock)
 - Desk length..... 90763120
 - Full length..... 90763121
- 2 Arm pad, pair (includes two blue arm pads and mounting screws)
 - Desk length..... 90763122
 - Full length..... 90763023
- 3 Spring lock (10 each) 90763024


CASTER

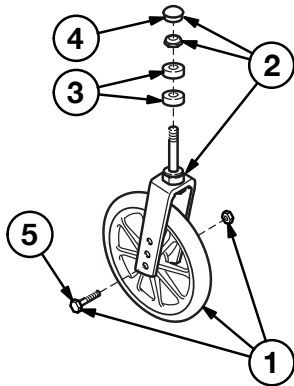

Complete caster (fork and wheel),
8" x 1" solid rubber tire..... 90763140

- 1 Caster wheel, axle and nut 90763141
- 2 Caster fork, stem nut and dust cap 90763042
- 3 Caster stem bearing (10 each) 90763043
- 4 Caster dust cap (10 each)..... 90763044
- 5 Caster axle and nut (10 each)..... 90763045

UPHOLSTERY (ALL SIDEFRAMES)

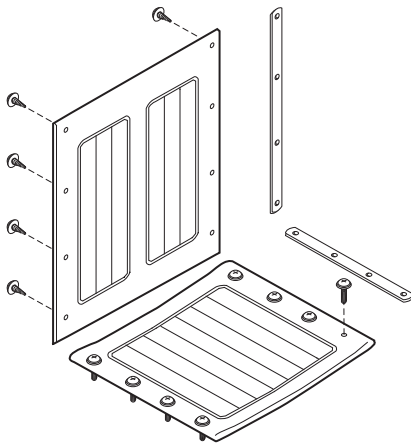


Back upholstery, blue leatherette, with
built-in chart holder (includes mounting
strips and hardware)

- 16" W..... 907631A1
- 18" W..... 907631A2
- 20" W..... 907631A3


Seat upholstery, blue leatherette
(includes mounting strips and
hardware)

- 16" W x 16" D..... 907631B1
- 18" W x 16" D..... 907631B2
- 20" W x 16" D..... 907631B3


FRONT RIGGING, DETACHABLE

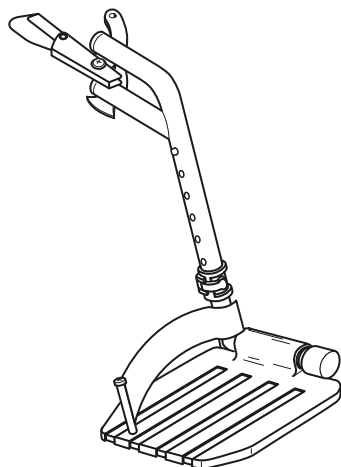

Footrest, pair, all widths
Includes right and left hanger with release
and aluminum footplate with extension
and heel loop 90763130

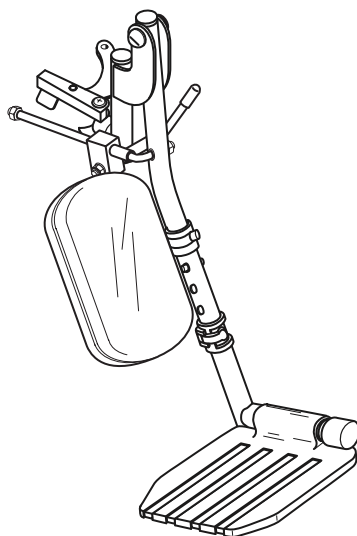


Low-pivot elevating legrest, pair, all widths
Includes right and left hanger with release,
aluminum footplate with extension, and blue
legrest panel (does not include
heel loop) 90763131

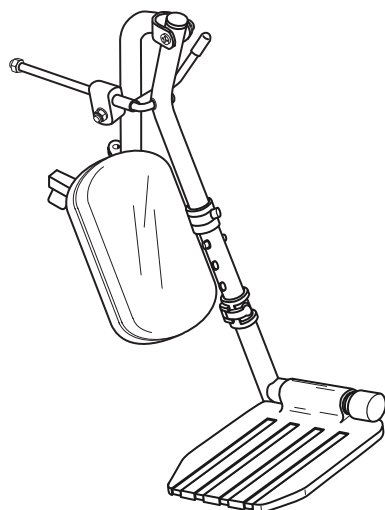

Panel kit for both low-pivot and
high-pivot elevating legrests
Includes 2 blue padded panels
and hardware 90763132

High-pivot elevating legrest, pair, all widths
Includes right and left hanger with release,
aluminum footplate with extension, and blue
legrest panel (does not include
heel loop) 90763135

REAR WHEEL

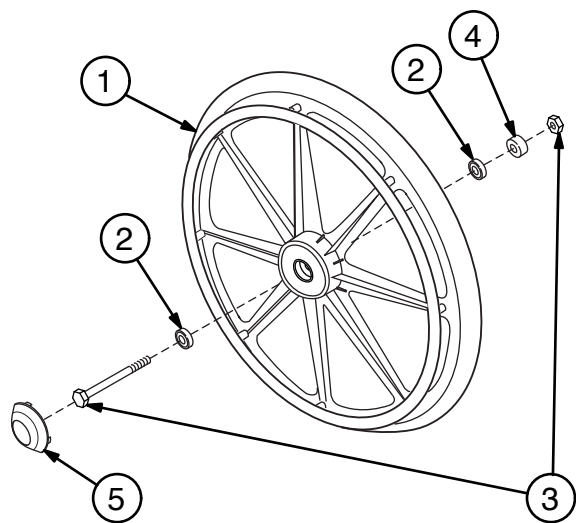

- | | | |
|---|--|----------|
| 1 | Rear wheel with handrim, 24" x 1", solid rubber tire (includes bearings; does not include axle or nut) | 90763155 |
| 2 | Wheel bearing (10 each) | 90763052 |
| 3 | Rear axle and nut (10 each) | |
| | Fixed-arm sideframe | 90763053 |
| | Detachable-arm sideframe | 90763054 |
| 4 | Spacer (10 each)..... | 90763055 |
| 5 | Hubcap for Traveler LX (10 each) | 90763156 |

ACCESSORIES

Traveler LX accessories (not shown)

- | | | |
|---|---|----------|
| 1 | Heel loop, pair..... | 90763064 |
| 2 | Wheel lock, push to lock, pair (includes rubber tips) | |
| | Fixed-arm chair | 90763170 |
| | Detachable-arm chair..... | 90763160 |
| 3 | Rubber tip, wheel lock (10 each)..... | 90763063 |
| 4 | Wheel lock extension, pair | 90763062 |
| 5 | Positioning belt (metal buckle) | 92133040 |
| 6 | Anti-Fold bar | 92523850 |
| 7 | O2-IV holder | B10320F |
| 8 | O2 cylinder holder..... | B10325 |
| 9 | Anti-Tippers (pair) | |
| | 20" seat height | 90763065 |
| | 18" seat height | 90763066 |

Everest & Jennings®

**GF Health Products, Inc.
2935 Northeast Parkway
Atlanta, Georgia 30360 USA**

**tel: 770-368-4700
fax: 770-368-2386**



GRAHAM-FIELD
www.grahamfield.com

**Traveler LX Parts Catalog
GF1400015RevB15**