

Tools required: Phillips screwdriver

- ⚠ **WARNING: Important! Read and understand these instructions before installing the Rehab Shower Commode Seat Hooks. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the Rehab Shower Commode Seat Hooks are not properly assembled and installed, personal injury and damage to the Rehab Shower Commode could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Everest & Jennings replacement parts. Non-Everest & Jennings replacement parts could cause personal injury and damage to the Rehab Shower Commode.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

Install Hooks

1. Place the seat, cushioned side down, on a flat surface.
2. Assemble a seat hook to each corner of the seat as shown in figure 1. The L-hook must be located as shown in figure 1.
3. Tighten the screws securely to prevent the hooks from sliding or turning.

Install Seat

1. Ensure the seat stops are positioned on both seat rails as shown in figure 2 to prevent the seat from sliding forward. Each screw and nut should be tight enough to prevent the seat stop from moving without deforming the end of the seat rail, which would prevent easy folding of the Rehab Shower Commode.
2. Open the commode to its full width as shown in figure 3.
3. Place the seat on the seat rails as shown in figure 3 **with the curved seat edge at rear**.
4. Hook the side with two J-hooks on the seat rails with the front hook located **behind** the seat stop. Lower the other side of the seat until it rests on the seat rails.
5. Lift the seat rails about an inch to allow the remaining hooks to engage the seat rails, then push down on the seat to open the chair to its full width and lock the seat in place. **Ensure front hooks are butted against backs of seat stops to prevent the seat from sliding forward.**

Remove Seat

1. Lift the seat rails about an inch to narrow the chair and disengage the seat hooks, then lift the seat out of the chair.
2. The chair can now be folded completely.

- ⚠ **WARNING: Ensure the Rehab Shower Commode seat is installed as described above before use, or personal injury and damage to the Rehab Shower Commode could result.**

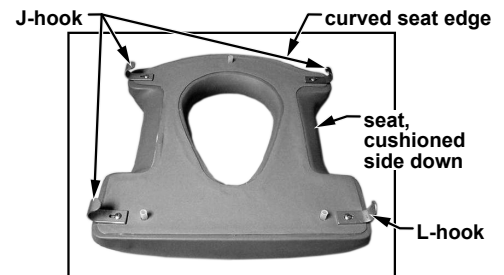


figure 1

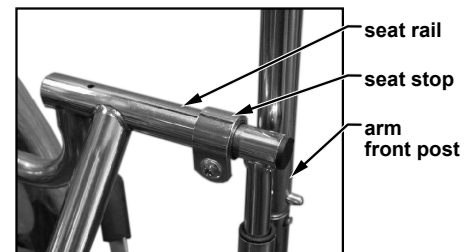


figure 2

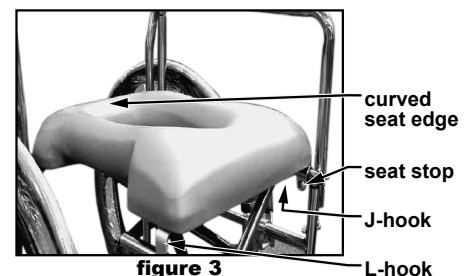


figure 3



figure 4



GRAHAM-FIELD
www.grahamfield.com

GF Health Products, Inc. is not responsible for typographical errors. All illustrations, specifications, packaging and warranties contained in this document are based on the latest product information available at the time of printing. The most current product information, including the most current revision of this manual, can be found online at www.grahamfield.com. Everest & Jennings, E&J, and Graham-Field are registered trademarks of GF Health Products, Inc.

© 2010 GF Health Products, Inc. • 2935 Northeast Parkway • Atlanta, GA 30360 • tel 770-368-4700

GF1000001RevB14