

Tools required: Phillips screwdriver

- ⚠ WARNING: Important! Read and understand these instructions before using the Freedom Walker. Do not use this product without proper instruction from a healthcare professional. If the Freedom Walker is not properly assembled and used, personal injury and damage to the Freedom Walker could result.**
- ⚠ WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. The use of non-Lumex replacement parts could cause personal injury, property damage, and void the warranty to the Freedom Walker.**
- ⚠ WARNING: This product has a maximum weight capacity of 300 lb (140 kg), EVENLY DISTRIBUTED.**
- ⚠ WARNING: Always use caution when using this product on wet or slippery surfaces.**
- ⚠ WARNING: GF Health Products, Inc. ("GF") assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

INTENDED USE

The Freedom Walker is intended as a walking aid.


UNPACKING

1. Check for obvious damage to the carton or its contents. If damage is evident, please notify the carrier and your Graham-Field authorized distributor.
2. Remove all loose packing from the box.
3. Carefully remove partially assembled walker from shipping container. Each box should contain the following components:

Contents

Quantity	Description
1	Partially assembled walker frame, outriggers, adjustable seat and gate
1	PVC back section with mesh vinyl backrest sling
4	Caster (2 braking, non-braking)
2	Parallel arms
4	Wire lock pin (to secure adjustable height section)
1	Adjustable swing gate strap
14	Screw, 1/2" stainless steel (2 each for back section, 4 each for caster section, 8 each for outriggers)
6	Screw, 1" stainless steel (4 each for adjustable arm section, 2 each for swing gate)

ASSEMBLY INSTRUCTIONS

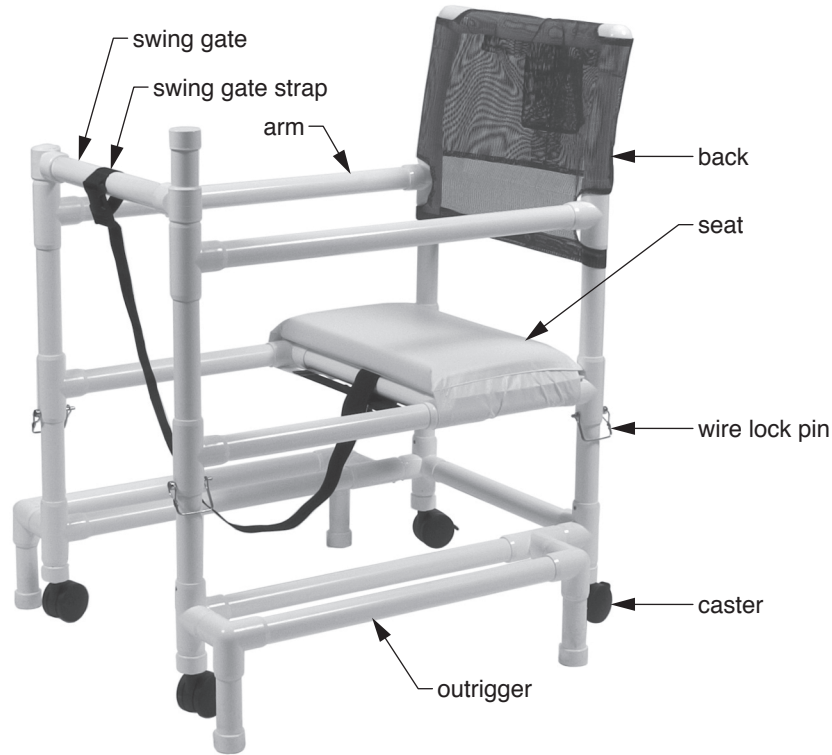

80500 Freedom Walker

Perform the following steps in the order listed. Refer to picture above.

Assemble Casters and Outriggers

1. Install one caster into each of the female fittings on the bottom frame section.
2. Secure each caster by inserting one 1/2" screw into the pre-drilled pilot hole on the female fitting. Tighten with a Phillips screwdriver.
3. Insert the outrigger into each side of the walker next to the casters: connect the front section of the outrigger (three-way fitting) into the front section (four-way fitting on the frame). Secure the outrigger with one 1/2" stainless steel screw. Insert the vertical support legs into the outrigger (4 each of leg piping). Secure each vertical support leg with one 1/2" stainless steel screw.

Assemble Adjustable Seat

1. Align bottom of the seat section fittings with the top of the 1" PVC pipe that extends out from the lower base near the casters. Slide the 1" PVC pipe vertically into the seat section until the last vertical adjustable hole is exposed. The four wire lock pins fit into each fitting that accommodates the 1" PVC pipe. The T fittings and the four four-way fittings take 1 wire lock pin each to secure the 1" PVC pipe from moving.
2. Connect the seat section with the bottom half of the 1" PVC pipe. The 1" PVC pipe needs to be secure with the fittings located above the casters.
3. Secure the 1" PVC pipe with one 1" stainless steel screw.

Assemble Parallel Arms

1. Facing the walker, insert the right side arm into the front section T fitting and back section four-way fitting. Insert the left side arm into the front section with the red strap into the T fitting, back section insert into the four-way fitting.
2. Secure both arms with one each 1" stainless steel screw.

Assemble PVC Back Section

Add the PVC back to the rear section of the arms. Secure the PVC back into the female fitting with one each 1" stainless steel screw.

Assemble Swing Gate

Includes one parallel pole (longest) and two vertical poles. The parallel pole serves as the main swing gate. The other two serve as pivot points for the swing gate.

1. Slide the parallel pole into the taller pivot pole. Facing the walker, insert the pivot pole into the T fitting on the right side. Secure the vertical pole with one 1" stainless steel screw.
2. Facing the walker, insert the shorter pivot pole into the T fitting into the left side. Secure with one 1" stainless steel screw.

⚠ WARNING: The red swing gate strap serves as additional security for the swing gate. The strap must be secured to the gate when the walker is in use.

Assemble Swing Gate Strap

The pre-assembled swing gate strap is attached to the base of the walker seat. Unroll the strap to finish assembly and loop the strap on the swing gate. Secure the swing gate with the red swing gate strap by snapping it to the swing gate.

The red swing gate strap must be on the right side of the walker.

⚠ WARNING: Ensure all screws and adjustable parts are securely installed before use.

MAINTENANCE AND CLEANING

⚠ WARNING: Inspect Freedom Walker weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your GF authorized distributor for repair or appropriate replacement parts.

▲ NOTICE: Do not use corrosive cleansers to clean this product.

Wipe weekly with a damp cloth; clean as needed with a mild soap and water solution.

LIMITED WARRANTY

SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a third party warrants a component, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted.

This limited warranty shall only apply to defects that are reported in accordance with the provisions set forth in this warranty document, within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. (See Obtaining Warranty Service below) This limited warranty is not transferable.

The warranted components and time periods are set forth below:

PVC Frame:.....one (1) year

All Other Materials:.....six (6) months

OBTAINING WARRANTY SERVICE

Customers located in the United States who wish to report a warranty issue, must contact GF directly by calling 1.770.368.4700 or by e-mailing a request to cs@grahamfield.com. Customers located outside the United States must contact the Distributor from whom they purchased the products. In both cases, further directions will be provided once the initial contact is made. This limited warranty shall only apply to defects that are reported within the applicable warranty period. Failure to abide by the specific directions will result in denial of the warranty claim.

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Matching of color, grain or texture except to commercially acceptable standards;
- 6) Changes in color caused by natural or artificial light;
- 7) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 8) Any labor or shipping charges incurred in the replacement part installation or repair;
- 9) Costs and expenses of regular maintenance and cleaning; and
- 10) Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. CERTAIN STATES MAY CONFER ADDITIONAL RIGHTS REGARDING WARRANTIES AND IN THOSE STATES GF'S LIABILITY AND THE LIABILITY OF GF'S SUPPLIERS, SHALL BE LIMITED TO THE FULLEST EXTENT PERMITTED BY LAW.


The warranties contained herein, together with GF's current Terms and Conditions, contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply. See GF's General Terms and Conditions on its website: www.grahamfield.com.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.

LUMEX®
by graham-field

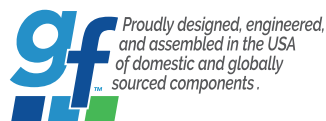

Manufactured For:
GF Health Products, Inc.
One Graham-Field Way, Atlanta, GA 30340
Made in U.S.A.



+1 770.368.4700

Information contained herein is subject to change without notice.
The most current and complete product information can be found on our website.
www.grahamfield.com

graham
field

© 2006, GF Health Products, Inc. All Rights Reserved.
Lumex is a registered trademark of GF Health Products, Inc.

LX_80500-INS-LAB-RevB23