Tools required: Phillips screwdriver (not included)

IMPORTANT SAFETY INFORMATION: PLEASE READ

- ⚠ WARNING: Important! Read and understand these instructions before assembling or using the Shower Chair. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the Shower Chair is not properly assembled and adjusted, personal injury and damage to the Shower Chair could result.**
- ⚠ WARNING: This product should not be used without proper instruction from a healthcare professional.**
- ⚠ WARNING: If components are damaged or missing, contact your Graham-Field authorized distributor immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the Shower Chair.**
- ⚠ WARNING: To avoid possible tilt hazard, do not leave user unattended. Attendant must always be behind and in control of Shower Chair when user is being positioned or Shower Chair is in use.**
- ⚠ WARNING: Do not operate the Shower Chair on streets or roadways.**
- ⚠ WARNING: Cancer and Reproductive Harm - www.p65warnings.ca.gov.**
- ⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**

INTENDED USE

The Shower Chair is intended for use by individuals who require seating support while showering, with the approval of a treating physician or authorized medical personnel.

Contraindications

- ⚠ WARNING: This product has the following maximum weight capacity:
89180 Series: 300 lb, EVENLY DISTRIBUTED.
89220 Series: 375 lb, EVENLY DISTRIBUTED.**

ASSEMBLY

Remove Shower Chair and components from the box. Place Shower Chair on a flat surface.

Info: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use Shower Chair. Contact carrier / dealer for further instruction.

Installing the Backrest Upholstery:

Fig. 1

Fig. 2

Fig. 3

1. Locate the backrest and backrest upholstery.
2. Lay the backrest on a flat surface.
3. Slide the backrest upholstery over the backrest tubing as shown in Figs. 1 and 2.
4. Attach the hook and loop over the top of the backrest as shown in Fig. 3.

Attaching the Backrest:

Fig. 4

Fig. 5

1. Lock the rear casters to prevent Shower Chair from moving during assembly: step on rear caster lever and press down to lock (to unlock: push up rear caster lever).
2. Locate the channel guides (2 each) on the back side of the chair frame near the seat. Slide the backrest support tubes into the channel guides as shown in Fig. 4.
3. Ensure the tubes are pressed all the way in and the backrest support tubes are even.
4. Locate the two Phillips head screws.
5. Insert the screws through backrest tube predrilled holes; use a Phillips screwdriver to tighten screws as shown in Fig. 5.

⚠ WARNING: Ensure backrest is securely installed and all hardware is tight before proceeding.

OPERATION

89180 Series

89220 Series

⚠ WARNING: Ensure the Shower Chair is assembled as described and toilet seat is in downward position before use. If the Shower Chair is not properly assembled and installed, personal injury and damage to the Shower Chair could result.

Center of Balance During Use

The point where the Shower Chair will tip forward, rearward or sideward is determined by its center of balance and stability, and the center of balance and stability is affected by the user's body position and weight distribution. If you fail to heed the following warnings, falling or tipping may occur and could cause severe personal injury.

⚠ WARNING: To avoid falling or tipping:

Have someone help you until you know the balance points of your Shower Chair and how to avoid tipping.

DO NOT put downward weight on one armrest at a time. If using the armrest for balance, hold both armrests at once to distribute your weight equally.

DO NOT put downward weight on the backrest.

DO NOT reach or lean if you must shift your weight to do so. Ask for help or use a device to extend your reach.

DO NOT try to reach an object if you must move forward in your seat so much that your buttocks do not stay in full contact with the seat, creating an unsafe position; your buttocks must remain in full contact with the seat at all times.

DO NOT lean backward to reach an object.

MAINTENANCE

⚠ WARNING: Ensure all Shower Chair components and attaching hardware are secure before each use.

To clean Shower Chair: use a mild soap and water solution, rinse with clear water, and then dry with a soft cloth. Ensure that Shower Chair is well rinsed before using.

▲ NOTICE: **DO NOT** use organic solvents, abrasive cleaners, or scouring pads on any part of Shower Chair.

LIMITED WARRANTY

SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, the following components are warranted for the time period set forth below:

Frame: one (1) year

All other components: six (6) months

The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1.770.368.4700, sending a fax request to 1.770.368.2386, or by e-mailing a request to cs@grahamfield.com. Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.

Manufactured For:

GF Health Products, Inc.
One Graham-Field Way, Atlanta, GA 30340
Made in China

© 2013, GF Health Products, Inc. All Rights Reserved.
Lumex is a registered trademark of GF Health Products, Inc.

89180-INS-LAB-RevE22

+1 770.368.4700

Information contained herein is subject to change without notice.
The most current and complete product information can be found on our website.
www.grahamfield.com

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© 2013, GF Health Products, Inc. Todos los derechos reservados.
Lumex es una marca de GF Health Products, Inc.