

- ⚠ **WARNING: Important! Read and understand these instructions before installing or using the Foot Deck Extension Kit. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the Foot Deck Extension Kit is not properly adjusted, personal injury and damage to the Foot Deck Extension Kit could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your GF Health Products, Inc. ("GF") authorized distributor immediately. DO NOT use substitute parts. Use only Basic American Medical Products replacement parts. Non-Basic American Medical Products replacement parts could cause personal injury and damage to the Foot Deck Extension Kit.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation or use of this product.**


FOOT DECK EXTENSION KIT INSTALLATION

- ⚠ **WARNING: Perform extension kit installation ONLY while the bed is unoccupied.**


1. Remove mattress retainer(s). Remove footboard and footboard mounting tubes (pinned to the main frame tubes) from the bed. Save hardware for reassembly.

Note: Footboards configured with Staff Control: disconnect Staff Control plug as shown at right *before* removing footboard.


Replace removed footboard mounting tubes with longer footboard mounting tubes provided with extension kit. Fasten footboard mounting tubes to footboard loosely at this time, to be tightened later.

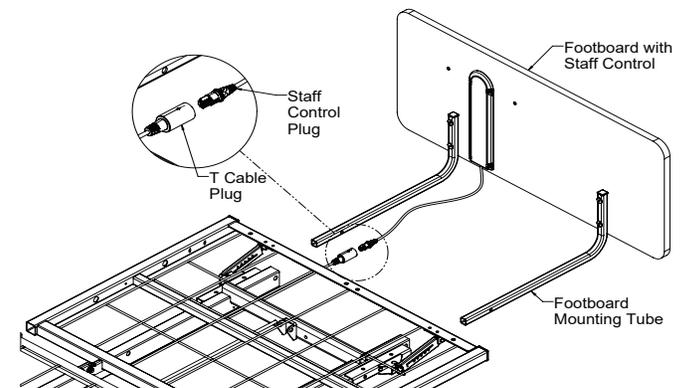


Step 1

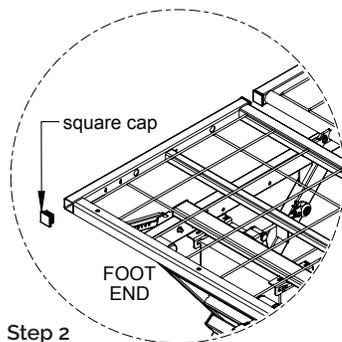

Step 2

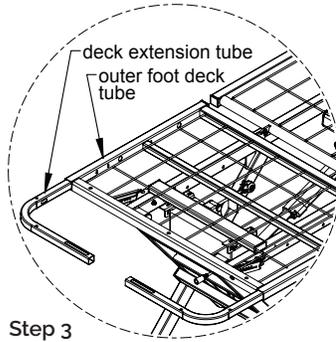


Step 3

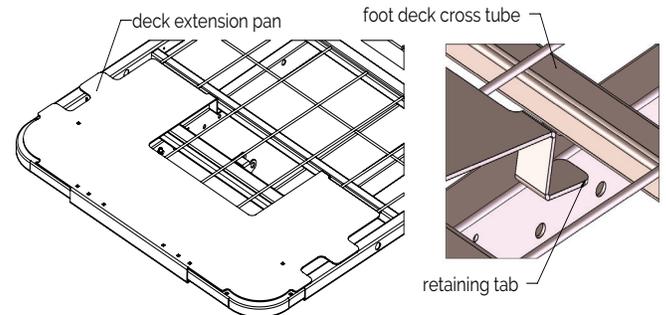

Step 4

2. Remove square caps from foot end of outside foot deck tubes as shown above. If bed has wide kit or Slide-W-I-D-E® attached, remove caps from foot end of outer tubes.
3. Orient and insert deck extension tubes into outer foot deck tubes as shown above with standard width deck, or to outermost tubes if mounting to wide kit or Slide-W-I-D-E.
4. Orient Deck Extension Pan on top of foot deck as shown above; then slide Deck Extension Pan toward head end with retaining tabs located under foot deck crossbrace as shown above.

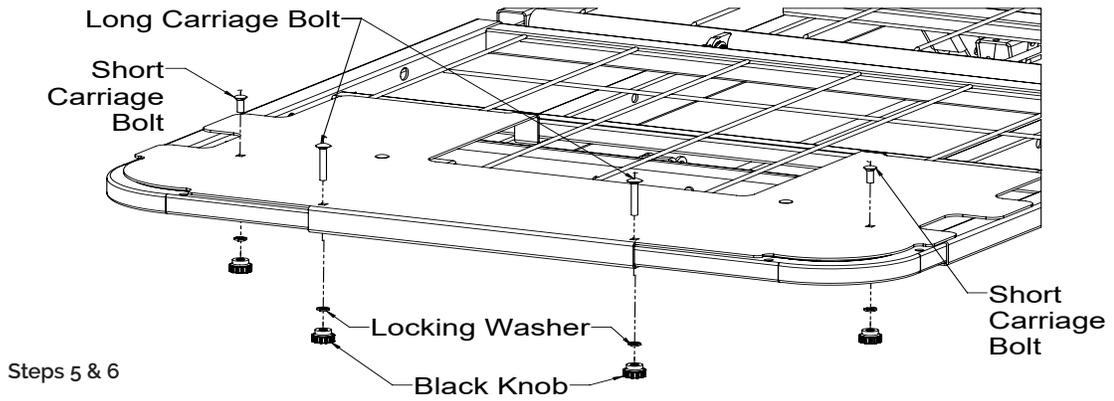

5. Align Deck Extension Pan holes with foot deck foot-end holes as shown above.


Note: All other models mount at both outside holes in locations shown above.

Secure Slide-Wide models as shown at above left with Short Carriage Bolt, Lock Washer, and Threaded Knob.

Secure all other models as shown at above right with Short Carriage Bolts, Lock Washers, and Threaded Knobs.

6. Secure Deck Extension Tubes to Deck Extension Pan as shown above.

7. Reinstall footboard using longer mounting tubes and pin in original 80" hole as shown at right. Replace mattress retainers.

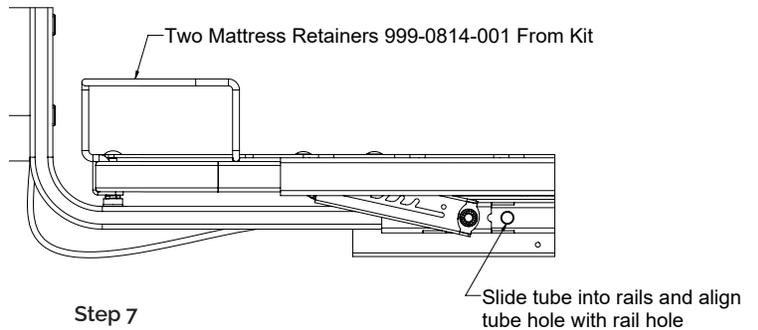

Note: In order to allow footboard mounting tubes to align properly and slide freely, only tighten footboard screws securely after footboard mounting tubes have been reinstalled and pinned into mainframe.

8. If configured with Staff Control, reconnect Staff Control and test all functions before use. Re-secure Staff Control to tube with cable tie.

⚠ WARNING: Before use, ensure all components are securely installed and Extension Kit is assembled as described above.



Manufactured By:
GF Health Products, Inc.
One Graham-Field Way, Atlanta, GA 30340
Made in USA

© 2022, GF Health Products, Inc. All Rights Reserved.
Basic American Medical Products is a trademark
of GF Health Products, Inc.
999-0957-115-RevB23



+1 770.368.4700

Information contained herein is subject to change without notice.
The most current and complete product information can be found on our website.
www.grahamfield.com

